

LEATHER BELTS–NON-LOCKING

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Including, but not limited to, securing our wrist and ankle restraints.



Wrist and ankle restraints sold separately.

INVENTORY ID:

ITEM	LENGTH	APPX. WEIGHT
B-218	18"	5 oz
B-224	24"	5 oz
B-230	30"	6 oz
B-236	36"	6 oz
B-242	42"	6 oz
B-248	48"	7 oz
B-254	54"	7 oz
B-260	60"	8 oz
B-266	66"	8 oz
B-272	72"	9 oz
B-278	78"	9 oz

INSTRUCTIONS: Feed the tail through the roller buckle to desired position and secure tail with the keeper. If using with a restraint, feed the tail through the restraint loop before feeding into the roller buckle.

Q: What size belt to order?

A: Distance from limb to anchor point multiplied by 2.

DESCRIPTION: Belts are 1 ¼" wide by specified length. The 18" belts have five size adjustments, 24" belts have ten size adjustments, and all other lengths have 12 size adjustments.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, and roller buckles.

MATERIALS USED: 12/13 oz leather, 1 ¼" steel roller buckle, and 1 ¼" steel keeper.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/25/16

POLYURETHANE BELTS—NON-LOCKING

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Including, but not limited to, securing our wrist and ankle restraints.



Wrist and ankle restraints sold separately.

INVENTORY ID:

ITEM	LENGTH	APPX. WEIGHT
B-218-POLY	18"	5 oz
B-224-POLY	24"	5 oz
B-230-POLY	30"	7 oz
B-236-POLY	36"	8 oz
B-242-POLY	42"	8 oz
B-248-POLY	48"	9 oz
B-254-POLY	54"	10 oz
B-260-POLY	60"	11 oz
B-266-POLY	66"	12 oz
B-272-POLY	72"	13 oz
B-278-POLY	78"	13 oz

INSTRUCTIONS: Feed the tail through the roller buckle to desired position and secure tail with the keeper. If using with a restraint, feed the tail through the restraint loop before feeding into the roller buckle.

Q: What size belt to order?

A: Distance from limb to anchor point multiplied by 2.

DESCRIPTION: Belts are 1 ¼" wide by specified length. The 18" belts have five size adjustments, 24" belts have ten size adjustments, and all other lengths have 12 size adjustments.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, and roller buckles.

MATERIALS USED: 1 ¼" polyurethane, 1 ¼" steel roller buckle, and 1 ¼" steel keeper.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/25/16


LEATHER BELTS-LOCKING

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Including, but not limited to, securing our wrist and ankle restraints.



Wrist and ankle restraints sold separately.

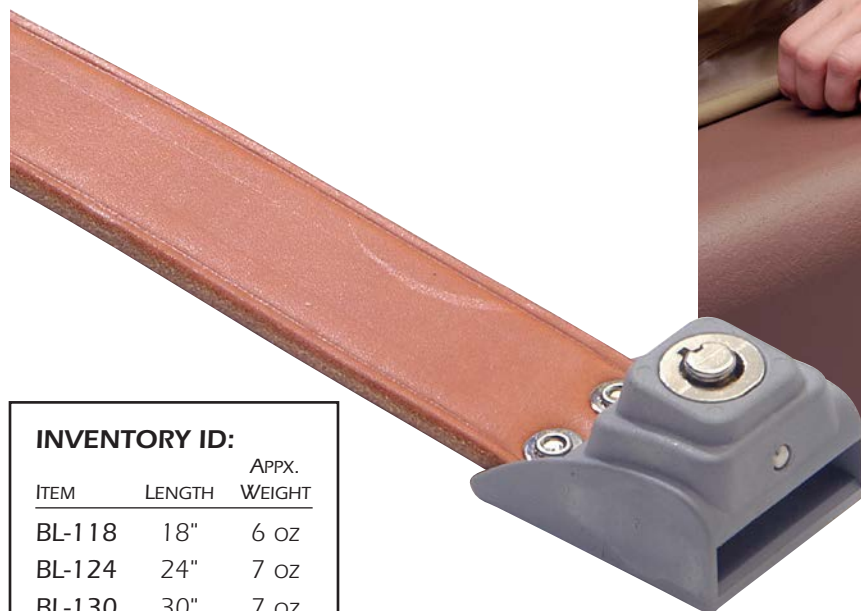

INVENTORY ID:

ITEM	LENGTH	APPX. WEIGHT
BL-118	18"	6 oz
BL-124	24"	7 oz
BL-130	30"	7 oz
BL-136	36"	8 oz
BL-142	42"	8 oz
BL-148	48"	9 oz
BL-154	54"	10 oz
BL-160	60"	11 oz
BL-166	66"	11 oz
BL-172	72"	12 oz
BL-178	78"	12 oz

INSTRUCTIONS: Feed the tail through the lock to desired position and follow locking instructions on the locks and keys spec sheet. If using with a restraint, feed the tail through the restraint loop before feeding into the lock.

Q: What size belt to order?

A: Distance from limb to anchor point multiplied by 2.

DESCRIPTION: Belts are 1¼" wide by specified length. The 18" belts have five size adjustments, 24" belts have ten size adjustments, and all other lengths have twelve size adjustments.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, and locks.

MATERIALS USED: 12/13 oz leather, Humane Restraint L-300 lock buckle.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/25/16

POLYURETHANE BELTS—LOCKING

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Including, but not limited to, securing our wrist and ankle restraints.



Wrist and ankle restraints sold separately.

INSTRUCTIONS: Feed the tail through the lock to desired position and follow locking instructions on the locks and keys spec sheet. If using with a restraint, feed the tail through the restraint loop before feeding into the lock.

Q: What size belt to order?

A: Distance from limb to anchor point multiplied by 2.

INVENTORY ID:

ITEM	LENGTH	APPX. WEIGHT
BL-118-POLY	18"	6 oz
BL-124-POLY	24"	7 oz
BL-130-POLY	30"	8 oz
BL-136-POLY	36"	9 oz
BL-142-POLY	42"	9 oz
BL-148-POLY	48"	10 oz
BL-154-POLY	54"	11 oz
BL-160-POLY	60"	12 oz
BL-166-POLY	66"	13 oz
BL-172-POLY	72"	14 oz
BL-178-POLY	78"	15 oz

DESCRIPTION: Belts are 1¼" wide by specified length. The 18" belts have five size adjustments, 24" belts have ten size adjustments, and all other lengths have twelve size adjustments.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, and locks.

MATERIALS USED: 1¼" polyurethane, Humane Restraint L-300 lock buckle.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/25/16

ONE PIECE SAFETY BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING




INTENDED USE: One piece safety belts may be used in many ways, such as controlling movement of a person in a bed, wheelchair, or stretcher.



INSTRUCTIONS: Lay the webbing strap flat so the bottom side of the buckle is showing. Wrap the strap around the person and object you wish to restrain them in, positioning so the bottom side of the buckle is against the person and the tongue of the buckle is accessible.

Pull up on the tongue of the buckle and feed the opposite end through the opening. Pull on the strap fed through to tighten, then push down on the tongue to secure. Be sure to push the tongue all the way down into the closed position or the webbing may slip through the opening. **NOTE:** Do not use for prolonged periods of time.

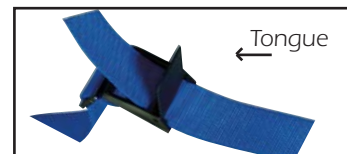

INVENTORY ID	APPROX. WEIGHT	LENGTH
29A-N48	10 oz.	48"
29A-N60	11 oz.	60"
29A-N72	12 oz.	72"
29A-N84	13 oz.	84"
29A-N96	14 oz.	96"
29A-N108	15 oz.	108"

COLORS AVAILABLE: Red, Orange, Yellow, Green, Blue, Gray, Black.

SAFETY CHECK: Including, but not limited to, webbing edges, and safety buckle.

MATERIALS USED: 2" webbing, and quick-action safety belt buckle.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.



Rev 4/8/05

RAIL SAFETY BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: Rail safety belts may be used in many ways, such as controlling movement of a person in a bed, wheelchair, or stretcher.

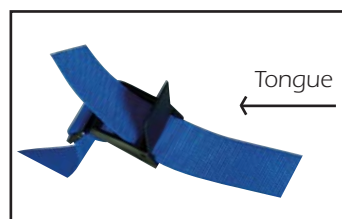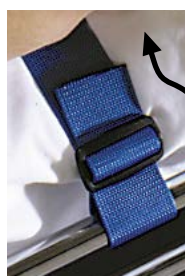

INSTRUCTIONS: Lay both pieces of webbing flat. Take the webbing piece with the buckle on one end and lay it on the bed, positioning the buckle in the middle of the bed. Loop the other end around the bar you wish to attach it to by first feeding under the bar and then over the top to reach the black slide.

Feed through the closest side of the slide bar over the top of the first layer. Pull tight to the bar and hold, then feed through the other side of the slide to secure. Then take the other webbing piece and place it on the other side of the bed positioning the end with the end tab in

the middle to feed through the buckle. Use the same procedure as previously listed to secure the strap to the proper bar.

Pull up on the tongue of the buckle and feed the angled end through the opening. Pull on the strap fed through to tighten, then push down on the tongue to secure. Be sure to push the tongue all the way down into the closed position or the webbing may slip through the opening. **NOTE: Do not use for prolonged periods of time.**

INVENTORY ID	APPROX. WEIGHT	LENGTH
--------------	----------------	--------

29N-40	... 11 oz	... 40"
--------	-----------	---------

29N-50	... 12 oz	... 50"
--------	-----------	---------

29N-60	... 13 oz	... 60"
--------	-----------	---------

COLORS AVAILABLE: Red, Orange, Yellow, Green, Blue, Gray, Black.

SAFETY CHECK: Including, but not limited to webbing edges, and safety buckle.

MATERIALS USED: 2" webbing, 2" acetal slide bars, and quick-action safety belt buckle.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/8/05

COTTON WEB GROMMET BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: These belts are designed with many uses in mind. They are very versatile and may be used in many situations such as a gait belt.

INSTRUCTIONS: Grommet belts close just like other roller buckles: pull end through buckle and insert tongue into appropriate size adjustment, then through keeper to hold the end. **NOTE:** Do not use for prolonged periods of time.

INVENTORY ID	LENGTH	SIZE ADJUSTMENTS
HRC-G4	4 Ft.	12
HRC-G5	5 Ft.	12
HRC-G6	6 Ft.	12

Other lengths are available upon request.

DESCRIPTION: Belts are 2" wide.

WEIGHT: Approx. weight of all belts is between 9 oz and 1 lb.

SAFETY CHECK: Including, but not limited to, grommets, roller buckle, stitching, and tips.

MATERIALS USED: #2 brass grommets, steel roller buckle, and cotton web.

CLEANING AND MAINTAINING: Dry clean or clean in your facility's laundry at less than 280°F. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store in a dry area to prevent mildew.

COTTON WEB BELTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: These belts are designed with many uses in mind. They are very versatile and may be used in many situations such as a gait belt.

INSTRUCTIONS: Undo the closure on the belt and lay flat. Decide how the belt will be best used, wrap around in a safe, secure manner, and pull ends toward each other. For closure, be sure the teeth on the buckle face you. Slide the metal-tipped end through the buckle from behind the teeth, pull to adjust sizing, then slide the end through hole on the other end of the buckle to secure.
NOTE: Do not use for prolonged periods of time.

INVENTORY ID

PASTEL STRIPE	BLUE STRIPE	LENGTH
HRC-PS4	HRC-BS4	4 Ft
HRC-PS5	HRC-BS5	5 Ft
HRC-PS6	HRC-BS6	6 Ft

Other lengths are available upon request.

DESCRIPTION: Belts are 2" wide.

WEIGHT: Approx. weight of all belts is between 3 oz and 8 oz.

SAFETY CHECK: Including, but not limited to, stitching, rivets, buckles, and tips.

MATERIALS USED: Cotton web, metal web buckle or spring buckle, and steel tips.

CLEANING AND MAINTAINING: Dry clean or clean in your facility's laundry at less than 280°F. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store in a dry area to prevent mildew.

WHEELCHAIR LAP BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: Helps hold seated patients safely in a wheelchair.



Easy Tie Knot



INSTRUCTIONS: Anchor each strap to wheelchair with Easy Tie Knot. Place patient in chair. Attach in front of patient with hook and loop, side release or Cam buckle.

INVENTORY ID	CLOSURE
HRC-LBCAM	CAM Buckle (A)
HRC-LBSR	Side Release Buckle (B)
HRC-LBHL	Hook & Loop (C)

DESCRIPTION: Belts are 2" wide.

WEIGHT: Approx. weight of all belts is 8 oz.

SAFETY CHECK: Including, but not limited to, buckle or hook and loop, polypropylene, and stitching.

MATERIALS USED: 2" polypropylene and 2" acetal cam buckle, 2" side release buckle or 1" hook and loop, and 2" acetal keepers with buckle belts only.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/8/05

HUMANE TRANSPORT SAFETY BOARD

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: For temporary restraint situations.

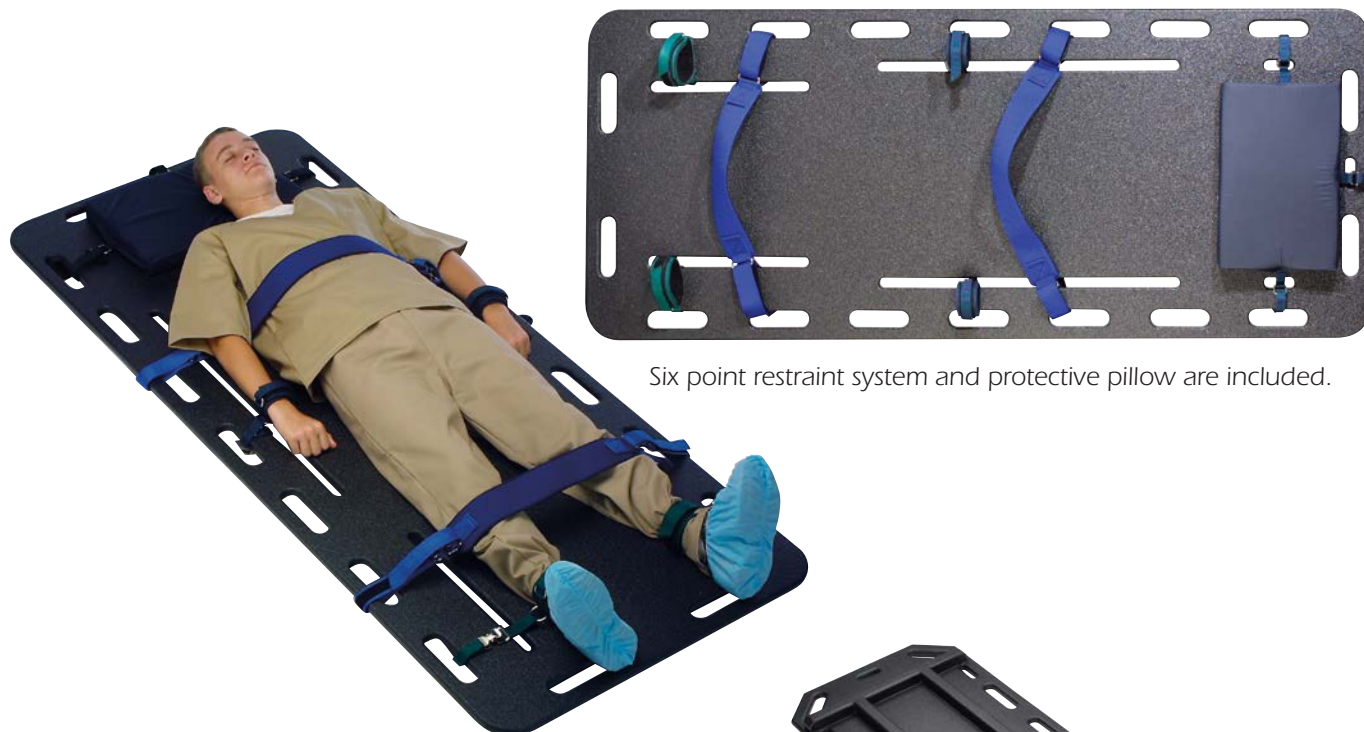

Six point restraint system and protective pillow are included.



Cross boards
prevent tipping.

Runner elevates
board to access handles.

INVENTORY ID	Size	Weight
--------------	------	--------

HST-100	32" w x 78" l	65 lbs
---------	---------------	--------

HST-200	24" w x 72" l	45 lbs
---------	---------------	--------

DESCRIPTION: 2½" tall (two 1½" runners on bottom for easy lifting); pillow; multiple handles. Restraints are included.

SAFETY CHECK: Including, but not limited to, runners, coating; restraint system checks — hook and loop, stitching, and polypropylene.

MATERIALS USED: 8-ply plywood, and sprayed on poly coating

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.


STORAGE: Store away from heat greater than 280°F or open flame.

Rev 3/28/16

NON-LOCKING WEB TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To secure a person's waist while in bed. It can be used with or without wrist and ankle restraints. Use only as specified in instructions.

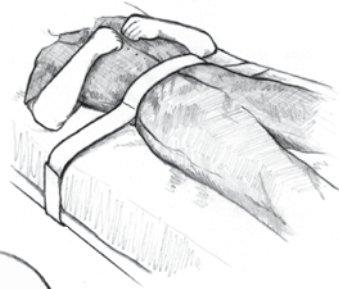

INSTRUCTIONS: Peel apart the hook and loop and lay the restraint flat on the bed. Attach the side straps to the bed frame and put hook and loop back on itself. Press down to seal the hook and loop. Position the person on the restraint so the waist belt will be comfortable when secure. Bring both ends of the waist belt together around the

person and secure by looping the tail end through the D-ring and put hook and loop back on itself. Press down to seal the hook and loop.
NOTE: Do not use for prolonged periods of time.

INVENTORY ID:

	Waist Size Range	Approx. Weight
2NL4S	24–36"	8 oz
2NL4M	30–42"	8 oz
2NL4L	36–48"	8 oz

DESCRIPTION: The side straps from lock to end are 40" long and 2" wide.

SAFETY CHECK: Including, but not limited to, hook and loop, stitching, and polypropylene webbing.

MATERIALS USED: 2" polypropylene web and 2" hook and loop.

CLEANING AND MAINTAINING: When cleaning, make sure all hook and loop is sealed on itself to prevent it from picking up materials in the laundry. Dry clean or launder in your facility's laundry at less than 280°F. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 8/20/04

WEB TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To secure a person's waist while in bed. It can be used with or without wrist and ankle restraints. Use only as specified in instructions.



INSTRUCTIONS: Unlock all locks and lay the restraint flat on the bed. Each side should have a lock near the edge of the bed and a third lock should be in the middle of the bed. Attach the side straps to the bed by looping the end of the strap around the bed frame and pull through the lock. Lock by using instructions on back of this page. Position the person on the restraint so the



waist belt will be comfortable when secure. Bring both ends of the waist belt together around the person and secure by looping the tail end through the lock. Continue by following locking procedures on back of this page.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID:

	Waist Size Range	Approx. Weight
30L4S	24-36"	8 oz
30L4M	30-42"	8 oz
30L4L	36-48"	8 oz

DESCRIPTION: The side straps from lock to end are 40" long and 2" wide.

SAFETY CHECK: Including, but not limited to, locks, stitching, and polypropylene webbing.

MATERIALS USED: Humane Restraint L-400 locks and 2" polypropylene web.

CLEANING AND MAINTAINING: Dry clean or launder in your facility's laundry at less than 280°F. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store away from heat greater than 280°F or open flame.




Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/3/05

WEB TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To secure a person's waist while in bed. It can be used with or without wrist and ankle restraints. Use only as specified in instructions.

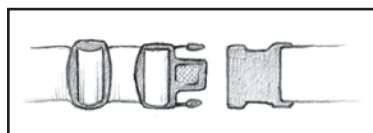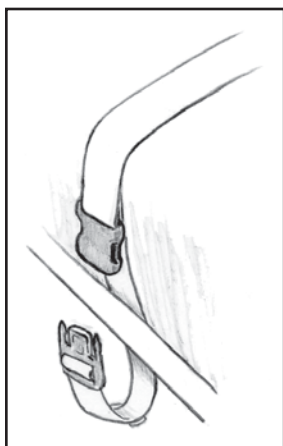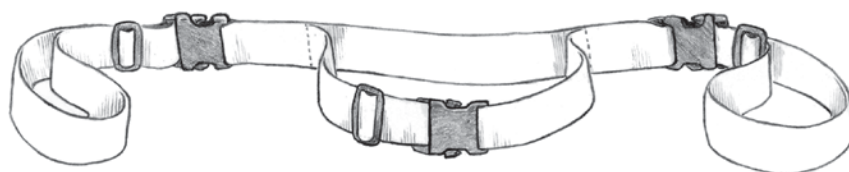

INSTRUCTIONS: Unbuckle the restraint and lay it flat on the bed. Each side should have a female buckle near the edge of the bed, a male end on the end of the strap, and a third buckle set should be in the middle of the bed. Attach the side straps to the bed by looping the end of the strap around the bed frame and snap the ends of

the buckle together. Position the person on the restraint so the waist belt will be comfortable when secure. Bring both ends of the waist belt together around the person and snap the ends of the buckle together.
NOTE: Do not use for prolonged periods of time.

INVENTORY ID:

	WAIST SIZE RANGE	APPROX. WEIGHT
304S	24-36"	7 oz
304M	30-42"	7 oz
304L	36-48"	7 oz

DESCRIPTION: The side straps from buckle to end are 40" long and 2" wide.

SAFETY CHECK: Including, but not limited to, buckles, stitching, and polypropylene webbing.

MATERIALS USED: Side release buckles and 2" polypropylene webbing.

CLEANING AND MAINTAINING: Dry clean or launder in your facility's laundry at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 9/6/02

LEATHER TORSO BELT SET

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To secure a person's waist while in bed. It can be used with or without wrist and ankle restraints. Use only as specified in the instructions.



INSTRUCTIONS: Apply the side straps to the bed by placing the loop end on top of the bed and the strap end (with holes) over the side of the bed. Attach the strap around the bed frame and pull through the lock. Lock by following instructions on back of this page. Use the same procedure for the side strap on the other side. Then take the waist belt, hold lock



face down, and pull the tail through loops of both side straps. Bring both ends of the waist belt together around the person and secure by looping the tail end through the lock. Continue by following locking procedures on the locks and keys spec sheet. **NOTE:** Do not use for prolonged periods of time.



INVENTORY ID: S-101

WEIGHT: approx. 1 lb 15 oz

DESCRIPTION: Set includes one BL-154 which is 54" long and 1 1/4" wide, and has twelve size adjustments, and two side straps which are 48" long and 1 1/4" wide with twelve size adjustments.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, and locks.

MATERIALS USED: 12/13 oz leather, three Humane Restraint L-300 lock buckles, and steel keepers.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/16

POLYURETHANE TORSO BELT SET

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To secure a person's waist while in bed. It can be used with or without wrist and ankle restraints. Use only as specified in the instructions.



INSTRUCTIONS: Apply the side straps to the bed by placing the loop end on top of the bed and the strap end (with holes) over the side of the bed. Attach the strap around the bed frame and pull through the lock. Lock by following instructions on back of this page. Use the same procedure for the side strap on the other side. Then take the waist belt, hold lock



face down, and pull the tail through loops of both side straps. Bring both ends of the waist belt together around the person and secure by looping the tail end through the lock. Continue by following locking procedures on the locks and keys spec sheet. **NOTE:** Do not use for prolonged periods of time.



INVENTORY ID: S-101-POLY

WEIGHT: approx. 2 lb 3 oz

DESCRIPTION: Set includes one BL-154-POLY which is 54" long and 1 1/4" wide, and has twelve size adjustments, and two side straps which are 48" long and 1 1/4" wide with twelve size adjustments.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, and locks.

MATERIALS USED: Polyurethane, three Humane Restraint L-300 lock buckles, and steel keepers.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/4/05

LOCKING LEG WEIGHT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To allow a restrained person to walk but impedes the progress of the person wearing it. At no time should a restrained person be unattended or unobserved by law enforcement personnel. Use only as specified in the instructions.



Featuring
L-300 Lock

INSTRUCTIONS: Before applying the restraint, unlock and unwrap completely. Wrap the weight around the ankle and adjust to the proper fitting by use of the hook and loop straps. Feed the attached strap through the

lock, then lock using the instructions on the locks and keys spec sheet.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPROX. WEIGHT
Leg Weight	TW-100	10 lb 6 oz each
	TW-200	20 lb 8 oz each

DESCRIPTION: The 10 lb. weight pouch is 14" long and 9" wide. The 20 lb. weight pouch is 20" long and 13" wide. The attached locking leather strap is 27" long and 1 1/4" wide with 14 adjustments. The two nylon straps are 26" long and 1" wide. Hook and loop is on the last 16" of the nylon straps.

SAFETY CHECK: Including, but not limited to stitching, rivets, lock, hook and loop, nylon, and steel loops.

MATERIALS USED: Black dyed 11+ oz leather, 1" welded steel loops, 1" black nylon, 1" hook and loop, L-300 lock, and Elmer's leg weight.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontamination blood-borne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Rev 4/21/16

POLYPROPYLENE BED RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To restrict motion of the limbs of the restrained person. Fifth and sixth point restraints are also available to further restrict movement. Use only as specified in the instructions.



INSTRUCTIONS: Check whether you have a wrist or ankle bed restraint; wrist restraints are blue and ankle restraints are green. Remember, a person raises their hands to the blue sky and walks on the green grass.

Open all closures, lay the restraint flat on the bed with both sides hanging off the side. Loop one side around the bed frame and seal the hook and loop by pressing it. Use the same procedure for the other side.

With cuffs open and flat, put the wrist or ankle on the cuff over the top and seal the hook and loop. Then place the outside end of the cuff over the top and seal the hook and loop by pressing it. Place the secondary strap over the top of the cuff and loop it through the D-ring. Pull the tail of the secondary strap back over the cuff and seal the second layer of hook and loop, again by pressing. Use the same procedure for the other limb.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID:

CLOSURE	WRIST	ANKLE
Side Release Buckle	NBW-101	NBA-101
Hook and Loop	NBW-104	NBA-104
L-400 Lock	NBLWL-403 . . .	NBLAL-403

WEIGHT: All weigh approx. 1 lb.

DESCRIPTION: NBW-101, NBA-101, NBLWL-403 and NBLAL-403: Standard size from side buckle to side buckle is 42" long by 2" wide, with side straps being 38" long. Wrist restraints are 12" long and 2" wide, and are 28" apart. Ankle restraints are 14" long and 2" wide and are spaced 14" apart.

NBW-104 and NBA-104: Standard size from side strap to side strap is 33" long by 2" wide, with side straps being 43" long. Wrist restraints are 13" long and 2" wide, and

are 26" apart. Ankle restraints are 15" long and 2" wide and are 14" apart.

Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, polypropylene, stitching, secondary closures, and hook and loop.

MATERIALS USED: Hook and loop OR side release buckles OR L-400 locks, plastic slide bar OR steel D-rings.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F with the hook and loop in closed position. Hook and loop may be cleaned with a lint brush. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame. Store hook and loop in a closed position.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/16

POLYPROPYLENE TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Side Release Buckle
NBT-101



Hook and Loop
NBT-104



L-400 Lock
NBTL-401

INTENDED USE: To restrain a person while in bed.
May be used with or without wrist and ankle restraints.



INSTRUCTIONS: To use the polypropylene torso restraint, lay the restraint over the portion of the body which is to be restrained. If you are using as a chest restraint, lay the restraint over the chest but under the arms. If used as a sixth point, it should be placed above or below the knees. Loop the tail of the side strap under and around the bed frame, and snap the buckle closed or lock with instructions on back of this page, or seal hook and loop by pressing.
Note: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID	Approx. Weight
Side Release Buckle NBT-101	14 oz.
Hook and Loop NBT-104	14 oz.
L-400 Lock NBTL-401	14 oz.

DESCRIPTION: Polypropylene torso restraint is 96" long and 2" wide. The 2" side release buckle, L-400 lock or 2" hook and loop material is sewn 28" from each end. Custom sizing is available upon request.

SAFETY CHECK: Including, but not limited to, polypropylene, stitching, and side release buckles, L-400 lock buckles or hook and loop.

MATERIALS USED: 2" black nylon, 2" side release buckles or L-400 lock buckle or hook and loop, and 2" slide bar.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 400°F or open flame. Hook and loop should be stored in a closed position.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/20/16

LEATHER NON-LOCKING BED RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.

Wrist and ankle bed restraints shown with torso restraint.



INSTRUCTIONS: Check whether you have a wrist or ankle bed restraint. W stands for wrist, A for ankle. Another way to differentiate is that the wrist cuffs are 26" apart on the bed restraint and the ankle cuffs are 14" apart.

Lay the restraint flat on the bed with the straps hanging off the side. Loop the straps around the anchoring point and pull through the roller buckle and keeper. Use the same procedure for the strap on the other side.

Then open the restraint by pulling out the leather strap on the wrist or ankle cuff so it is no longer through the loop. Pull up on the end to release from the loop and lay flat. Put the wrist or ankle on the flattened restraint, wrap the cuff around the limb, and adjust to the proper size. Feed the end of the attached strap through the loop to secure.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPX. WEIGHT
Wrist	CBW-201	2 lb 2 oz
Ankle	CBA-201	2 lb 3 oz

DESCRIPTION: The center webbing is 38" long and 4" wide. The leather straps are 32" long and 1 1/4" wide with twelve size adjustments. Wrist cuffs are 12" long and 3" wide with four size adjustments ranging from 8" – 10" and ankle cuffs are 14" long and 3" wide with four size adjustments ranging from 9 1/2" – 12". Custom sizing available upon request.

SAFETY CHECK: Including but not limited to, stitching, loops, rivets, roller buckles, and leather.

MATERIALS USED: 12/13 oz leather, light orthopedic leather lining, 3-ply cotton webbing, steel loops, and 1 1/4" steel roller buckles.

CLEANING AND MAINTAINING: For regular cleaning, wipe the leather with a mild soap and warm water. The webbing can be scrubbed with a soft brush and the same solution. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/4/05

POLYURETHANE NON-LOCKING BED RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

Wrist and ankle bed restraints shown with torso restraint.



INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.



INSTRUCTIONS: Restraints are color coded for quick visual identification. Wristlets are blue and anklets are green. Remember, a person raises their hands to the blue sky and walks on the green grass. Another way to tell is that the wristlets are spaced 26" apart and the anklets are 14" apart.

Lay the restraint flat on the bed with the straps hanging off the side. Loop the straps around the anchoring point and pull through the roller buckle and keeper. Use the same pro-cedure for the strap on the other side.

Then open the restraint by pulling out the polyurethane strap on the wrist or ankle cuff so it is no longer through the loop. Pull up on the end to release from the loop and lay flat. Put the wrist or ankle on the flattened restraint, wrap the cuff around the limb, and adjust to the proper size. Feed the end of the attached strap through the loop to secure.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPX. WEIGHT
Wrist	CBW-201-POLY	2 lb 2 oz
Ankle	CBA-201-POLY	2 lb 3 oz

DESCRIPTION: The center section is 38" long and 3" wide. The polyurethane side straps are 32" long and 1 ¼" wide with twelve size adjustments. Wrist cuffs are 12" long and 2 ¾" wide with four size adjustments ranging from 8" – 10" and ankle cuffs are 14" long and 2 ¾" wide with four size adjustments ranging from 9 ½" – 12". Wristlets are spaced at 26" and anklets are 14" apart. Custom sizing available upon request.

SAFETY CHECK: Including but not limited to, loops, rivets, roller buckles, and polyurethane.

MATERIALS USED: Polyurethane, steel loops, and 1 ¼" steel roller buckles.

CLEANING AND MAINTAINING: For regular cleaning, wipe the polyurethane with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/4/05

LEATHER LOCKING BED RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. The lock promotes authorized release only. Use only as specified in the instructions.



Wrist and ankle
bed restraints shown
with two torso restraints.



INSTRUCTIONS: Check whether you have a wrist or ankle bed restraint. W stands for wrist, A for ankle. Another way to differentiate is that the wrist cuffs are 26" apart on the bed restraint and the ankle cuffs are 14" apart. Lay the restraint flat on the bed with the straps hanging off the side. Loop the straps around the anchoring point and pull through the lock and keeper. Follow locking instructions on the locks and keys spec sheet. Use the same procedure for the strap on the other side.

Unlock cuffs using locking instructions on the locks and keys spec sheet, open, and lay flat. Wrap the cuff around the limb and adjust to the proper fitting. Follow locking instructions on the locks and keys spec sheet. Use the same procedure for the other limbs.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPX. WEIGHT
Wrist	CBLWL-203	2 lb 3 oz
Ankle	CBLAL-203	2 lb 5 oz

DESCRIPTION: The center webbing is 38" long and 4" wide. The leather straps are 32" long and 1 1/4" wide with twelve size adjustments. Wrist cuffs are 12" long and 2 1/2" wide with seven size adjustments. Ankle cuffs are 14" long and 2 1/2" wide with seven size adjustments. Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, stitching, loops, rivets, Humane Restraint locks, and leather.

MATERIALS USED: 12/13 oz leather, light orthopedic leather lining, 3-ply cotton webbing, steel loops, Humane Restraint L-300 locks.

CLEANING AND MAINTAINING: For regular cleaning, wipe the leather with a mild soap and warm water. The webbing can be scrubbed with a soft brush and the same solution. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/6

POLYURETHANE LOCKING BED RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. The lock promotes authorized release. Use only as specified in the instructions.



Wrist and ankle
bed restraints shown
with two torso restraints.



INSTRUCTIONS: Restraints are color coded for quick visual identification. Wristlets are blue and anklets are green. Remember, a person raises their hands to the blue sky and walks on the green grass. Another way to tell is that the wristlets are spaced 26" apart and the anklets are 14" apart.

Lay the restraint flat on the bed with the straps hanging off the side. Loop the straps around the anchoring point and pull through the lock and keeper. Follow locking

instructions on the locks and keys spec sheet. Use the same procedure for the strap on the other side. Unlock cuffs using instructions on the locks and keys spec sheet, open, and lay flat. Wrap the cuff around the limb and adjust to the proper fitting. Follow locking instructions on locks and keys spec sheet. Use the same procedure for the other limbs.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPX. WEIGHT
Wrist	CBLWL-203-POLY	2 lb
Ankle	CBLAL-203-POLY	2 lb 2 oz

DESCRIPTION: The center section is 38" long and 3" wide. The polyurethane side straps are 32" long and 1 1/4" wide with twelve size adjustments. Wrist cuffs are 12" long and 2 1/2" wide with seven size adjustments. Ankle cuffs are 14" long and 2 1/2" wide with seven size adjustments. Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, loops, rivets, Humane Restraint locks, and polyurethane.

MATERIALS USED: Polyurethane, steel loops, Humane Restraint L-300 locks.

CLEANING AND MAINTAINING: For regular cleaning, wipe the polyurethane with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/16

LEATHER WRIST-WAIST RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To secure a person's wrists at the waist. This restraint allows no freedom of movement of the arms. Use only as specified in the instructions.



Shown with
ADL-305
ankle cuffs.



We recommend
using this
restraint with
our disposable
fleece liners
FWL-200.



INSTRUCTIONS: Unlock all locks. Wrap belt around waist so that the lock is in back and on the right side of the body. Loop the end of the belt through the lock and follow L-300 locking procedures on the locks and and keys spec sheet to secure.

Then take one wrist and put it into the open wrist cuff, wrap the cuff around the wrist, and fit to the appropriate size adjustment. Follow L-300 locking procedures on the locks and keys spec sheet to secure the lock. Use this same procedure to secure the other wrist.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: MND-101

WEIGHT: approx. 1 lb 5 oz.

DESCRIPTION: The belt is 48" long and 1¼" wide with ten size adjustments and the cuffs are 12" long and 2½" wide with seven size adjustments.

SAFETY CHECK: Including, but not limited to, locks, rivets, stitching, and leather.

MATERIALS USED: Steel loop, Humane Restraint L-300 locks, 12/13 oz leather, light orthopedic leather lining, and rivets.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/25/16


POLYURETHANE WRIST-WAIST RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To secure a person's wrists at the waist. This restraint allows no freedom of movement of the arms. Use only as specified in the instructions.



Shown with
ADL-305
ankle cuffs.

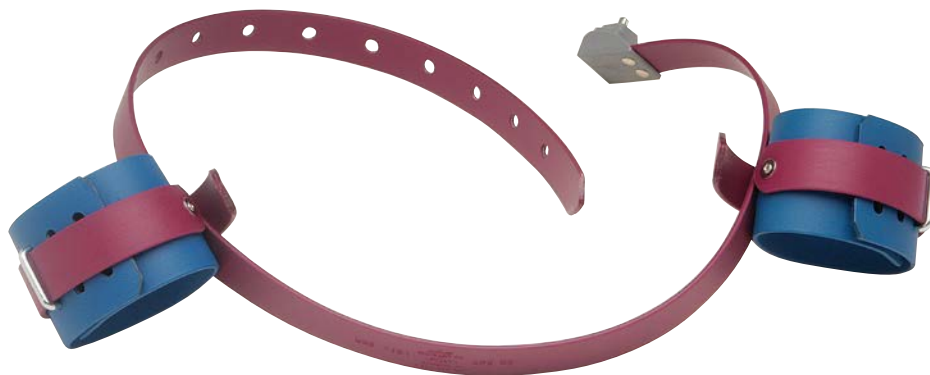

We recommend
using this
restraint with
our disposable
fleece liners
FWL-200.



INSTRUCTIONS: Unlock all locks. Wrap belt around waist so that the lock is in back and on the right side of the body. Loop the end of the belt through the lock and follow L-300 locking procedures on the locks and keys spec sheet to secure.

Then take one wrist and put it into the open wrist cuff, wrap the cuff around the wrist, and fit to the appropriate size adjustment. Follow L-300 locking procedures on the locks and keys spec sheet to secure the lock. Use this same procedure to secure the other wrist.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: MND-101-POLY

WEIGHT: approx. 1 lb 3 oz.

DESCRIPTION: The belt is 48" long and 1 1/4" wide with ten size adjustments and the cuffs are 12" long and 2" wide with eight size adjustments from 6" to 9 3/4".

SAFETY CHECK: Including, but not limited to, locks, rivets, and polyurethane.

MATERIALS USED: Steel loop, Humane Restraint L-300 locks, and polyurethane.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/16

LEATHER WRIST-WAIST RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Shown with
ADL-305
ankle cuffs.

INTENDED USE: This restraint has the ability to restrain the wrists against the body or up to 12" from the waist; each arm can be adjusted separately. Use only as specified in the instructions.



We recommend
using this
restraint with
our disposable
fleece liners
FWL-200.



INSTRUCTIONS: Unlock all locks. Wrap belt around waist so that the lock is in back and on the right side of the body. Loop the end of the belt through the lock and follow L-300 locking procedures on locks and keys spec sheet to secure. Adjust the leather side straps to the desired length of movement and use the same locking procedures to secure the straps.

Then take one wrist and put it into the open wrist cuff, wrap the cuff around the wrist, and fit to the appropriate size adjustment. Follow L-300 locking procedures on locks and keys spec sheet to secure the lock.

Use this same procedure to secure the other wrist.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: MNDL-401

WEIGHT: approx. 1 lb 15 oz

DESCRIPTION: This restraint is a combination of a Humane Restraint belt and a pair of WA-501 wrist cuffs. The belt is 48" long and 1 1/4" wide with ten size adjustments and the cuffs are 12" long and 2 1/2" wide with seven size adjustments. This restraint also has two 12" straps that secure the wrist cuffs to the belt.

SAFETY CHECK: Including, but not limited to, locks, rivets, loops, stitching, and leather.

MATERIALS USED: Steel loops, Humane Restraint L-300 locks, 12/13 oz leather, light orthopedic leather lining and rivets.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.


Rev 4/25/16

POLYURETHANE WRIST-WAIST RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Shown with
ADL-305
ankle cuffs.

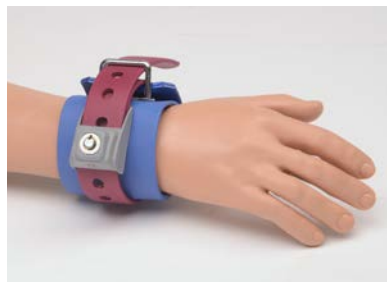

We recommend
using this
restraint with
our disposable
fleece liners
FWL-200.



INSTRUCTIONS: Unlock all locks. Wrap belt around waist so that the lock is in back and on the right side of the body. Loop the end of the belt through the lock and follow L-300 locking procedures on the locks and keys spec sheet to secure.

Then take one wrist and put it into the open wrist cuff, wrap the cuff around the wrist, and fit to the appropri-

ate size adjustment. Follow L-300 locking procedures on the locks and keys spec sheet to secure the lock. Use this same procedure to secure the other wrist.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: MNDL-401-POLY

WEIGHT: approx. 2 lb

DESCRIPTION: This restraint is a combination of a Humane Restraint belt and a pair of WA-501-POLY wrist cuffs. The belt is 48" long and 1 1/4" wide with ten size adjustments and the cuffs are 12" long and 2 1/2" wide with seven size adjustments. This restraint also has two 12" straps that secure the wrist cuffs to the belt.

SAFETY CHECK: Including, but not limited to, locks, rivets, loops, and polyurethane.

MATERIALS USED: Steel loops, Humane Restraint L-300 locks, polyurethane, and rivets.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/16

LEATHER BED TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrain a person while in a bed. It may be used with or without wrist and ankle restraints.

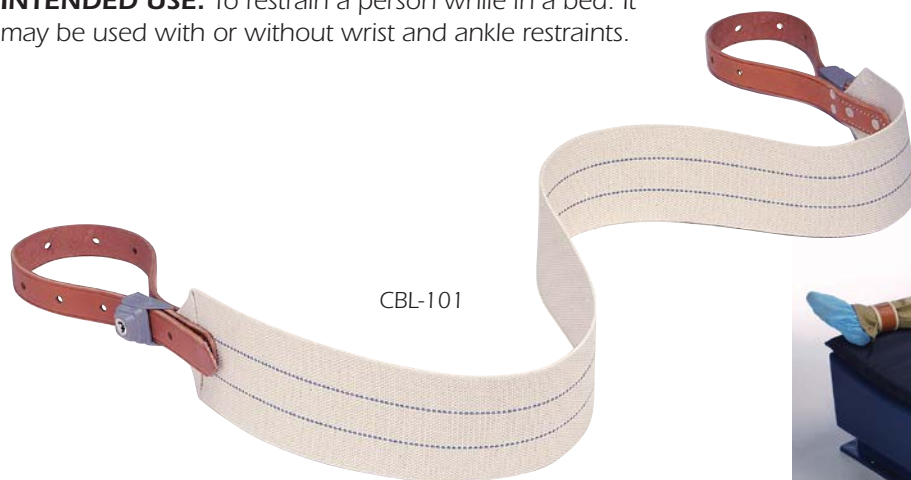

CBL-101

Torso restraint shown with wrist and ankle bed restraints.



Roller Buckle
CBR-101

INSTRUCTIONS: To use the CBR-101, lay the restraint over the portion of the body which is to be restrained. If you are using as a chest restraint, lay the restraint over the chest but under the arms. If you are using as a sixth point restraint, lay the restraint above or below the knees. Loop the tail of each leather strap under and around a bar or rail. Feed the tail through the roller buckle, positioning the tongue of the roller buckle in the appropriate size adjustment for both sides.

To use the CBL-101, position the restraint as in CBR-101. Loop the tail of each leather strap under and around a bar or rail. Feed the tail into the lock. Continue by using locking instructions on Locks and Keys spec sheet.

NOTE: Do not use for prolonged periods of time.



L-300 Lock
CBL-101

INVENTORY ID

APPX. WEIGHT

CBR-101 (roller buckles) 1 lb 3 oz

CBL-101 (L-300 locks) 1 lb 8 oz

DESCRIPTION: Center webbing is 48" long and 4" wide. Each side strap is 20" long and 1 1/4" wide. The leather straps have ten size adjustments. Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, webbing, and locks or buckles.

MATERIALS USED: Two-ply cotton webbing; welded steel keepers; 12/13 oz leather; 1 1/4" wide roller buckles or Humane Restraint L-300 lock buckles.

CLEANING AND MAINTAINING: For regular cleaning, wipe the leather with a mild soap and warm water. The webbing can be scrubbed with a soft brush and the same solution. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 3/30/16

POLYURETHANE BED TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrain a person while in a bed. It may be used with or without wrist and ankle restraints.

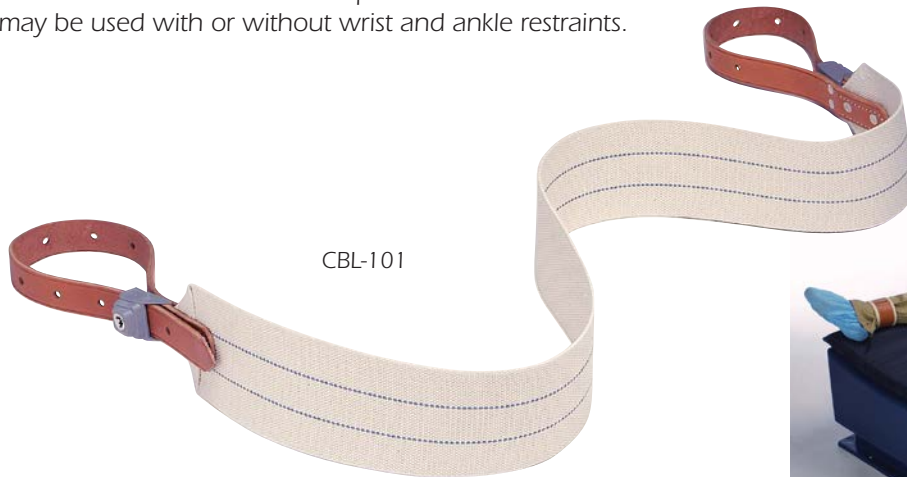

CBL-101

Torso restraints shown with wrist and ankle bed restraints.



Roller Buckle
CBR-101

INSTRUCTIONS: To use the CBR-101-POLY, lay the restraint over the portion of the body which is to be restrained. If you are using as a chest restraint, lay the restraint over the chest but under the arms. If you are using as a sixth point restraint, lay the restraint above or below the knees. Loop the tail of each polyurethane strap under and around bed frame. Feed the tail through the roller buckle, positioning the tongue of the roller buckle in the appropriate size adjustment for both sides.

To use the CBL-101-POLY, position the restraint as in CBR-101. Loop the tail of each polyurethane strap under and around bed frame. Feed the tail into the lock. Continue by using locking instructions on Locks and Keys spec sheet.

NOTE: Do not use for prolonged periods of time.



L-300 Lock
CBL-101

INVENTORY ID

APPX. WEIGHT

CBR-101-POLY (roller buckles) 1 lb 3 oz

CBL-101-POLY (L-100 locks) 1 lb 8 oz

DESCRIPTION: Center webbing is 48" long and 4" wide. Each side strap is 20" long and 1 1/4" wide. The polyurethane straps have ten size adjustments. Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, webbing, and locks or buckles.

MATERIALS USED: Two-ply cotton webbing, welded steel keepers, polyurethane, and 1 1/4" wide roller buckles or Humane Restraint L-300 lock buckles.

CLEANING AND MAINTAINING: For regular cleaning, wipe the polyurethane with a mild soap and warm water. The webbing can be scrubbed with a soft brush and the same solution. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.


Rev 3/30/16

TRANSPORT LEG BRACE

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To allow a restrained person to stand trial or fly while jurors or other passengers are unaware of the presence of any restraints. Although this leg brace will impede the progress of the person wearing it, at no time should the restrained person be unattended or unobserved by law enforcement personnel. Use only as specified in the instructions.

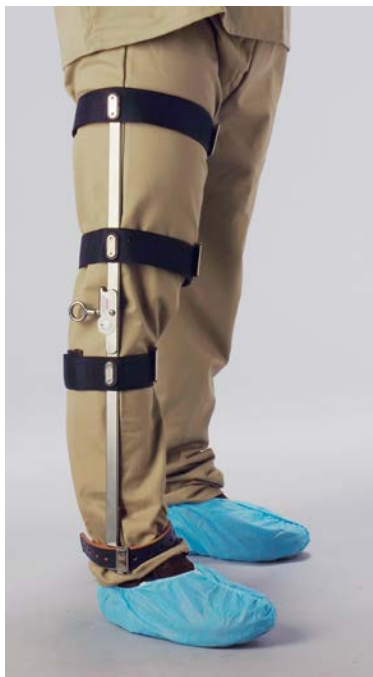

INSTRUCTIONS: First, make sure you are correctly using a right or left leg restraint. To verify, check the stamp inside the metal bar by lock. L is for left, R is for right.

After you have verified the correct side to apply the restraint, unlock the lock and pull apart all hook and loop straps. Then hold the restraint against the leg with the hinge at the knee, close the hook and loop straps around the leg, and lock the leather strap at the ankle (using the locking instructions on the lock and key spec sheet).

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: LAB-100-R (right), LAB-100-L (left)

WEIGHT: appx. 2.5 lb.

DESCRIPTION: The metal piece on the leg brace is 27" long. The 2" wide nylon straps from top to bottom are 37½", 32½", and 30½" in length. The leather strap at the bottom is 17" long and 1¼" wide with 10 size adjustments.

SAFETY CHECK: Including, but not limited to, lock, hook and loop, hinge, nylon edges, stitching, rivets, and metal tips.

MATERIALS USED: Black dyed 11+ oz leather, 2" metal tips, 2" black nylon, 1½" hook and loop, L-300 lock, and stainless steel upright with loop lever.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil. Be sure to oil the hinge and lever weekly to assure properly working parts.


STORAGE: Store in a dry area to prevent mildew.

Rev 4/20/16

POLYPROPYLENE LIMB HOLDERS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. The lock promotes authorized release only. Use only as specified in the instructions.

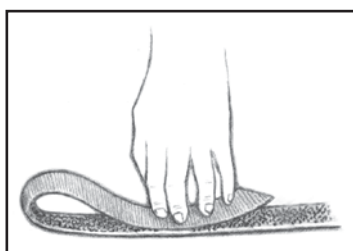

Pressing hook and loop.



INSTRUCTIONS: Open restraint completely and lay flat. Put the limb which is to be restrained on the flattened restraint and fold the sides over so the hook and loop are attached. Then feed the strap of the secondary closure through the loop and close the strap back onto itself;

fasten securely. Continue by wrapping the long strap around an anchor and feeding through the buckle; fasten securely.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPROX. WEIGHT
Wrist	NWL-101	15 oz per pair
Ankle	NAL-101	15 oz per pair

DESCRIPTION: Wrist cuffs are 12" long and 2" wide with 4" of hook and loop and are made of blue polypropylene for easy identification. Ankle cuffs are 14" long and 2" wide with 4" of hook and loop and are made of green polypropylene. Strap to secure to bed or rail is 48" long and 1½" wide.

SAFETY CHECK: Including, but not limited to stitching and hook and loop.

MATERIALS USED: Spring buckle, 2" nylon, 1½" hook and loop.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/20/16

POLYPROPYLENE LIMB HOLDERS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.



Hook & Loop
NWL-101 — Wrist (Blue)
NAL-101 — Ankle (Green)



L-400 Lock
NWL-104 — Wrist (Blue)
NAL-104 — Ankle (Green)



INSTR
coded
Wristle

green. Remember, a person raises their hands to the blue sky and walks on the green grass.

Open all closures and lay the restraint flat with additional straps to the left. Place the limb in the center of the cuff. Take the left side of the cuff and place it over the limb. Then take the right side of the cuff and



over the left side, sealing the loop by pressing.

Place the secondary strap (from the left) over the top of the cuff and loop it through the loop, or lock following instructions on back. Pull the tail of the secondary strap back over the cuff and seal the second layer of hook and loop by pressing.

Take the control strap and loop it under and around the anchor point. Then loop it through the teeth of

the buckle from the bottom, across the top of the buckle, and through the open section of the back of the buckle, or lock following instructions on back. Pull the tail end tight, place it back over itself and seal the hook and loop by pressing. Use the same procedure for the other limb. **NOTE:** Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	COLOR	APPROX. WEIGHT
Wrist, non-locking	NWL-101	Blue	1 lb. per pair
Wrist, locking	NWL-104	Blue	1 lb. per pair
Ankle, non-locking	NAL-101	Green	1 lb. per pair
Ankle, locking	NAL-104	Green	1 lb. per pair

DESCRIPTION: Wrist restraints are 13" long and 2" wide. Ankle restraints are 15" long and 1½ or 2" wide. Strap to secure to bed or chair is 40" long and 1" or 2" wide. Custom sizing is available upon request.

SAFETY CHECK: Including, but not limited to polypropylene, hook and loop and stitching.

MATERIALS USED: 1½" and 2" polypropylene, neoprene, 1" and 1½" hook and loop, steel loop and 1" steel spring buckle or L-400 lock.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. with hook and loop in closed position. Hook and loop may be cleaned with a lint brush. When decontaminating blood borne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame and with hook and loop in a closed position.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/21/16

HUMANE TETHER

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To provide a secure limited range of motion.
Use only as specified in the instructions.



INSTRUCTIONS:

Apply one end of the tether to the restrained person (i.e. wrist or on top of handcuffs), and the other to a secured object (i.e. wall mount or door mount). Proper handcuffing technique should be followed.

Note: Do not use for prolonged periods of time.

INVENTORY ID: WML-101 – Leather
WMR-101 – Polyurethane

WEIGHT: approx. 1 lb.

DESCRIPTION: Overall length 37". Center belt is 30" long and 1 1/4" wide.

SAFETY CHECK: Including, but not limited to, leather or polyurethane, stitching (leather only), rivets, and operation of handcuffs.

MATERIALS USED: 12/13 oz. leather or polyurethane, rivets, washers, steel loops and one pair of handcuffs.

CLEANING AND MAINTAINING: For regular cleaning, wipe with mild soap and warm water. After wiping the leather only, allow to dry and apply neatsfoot oil. When decontaminating bloodborne pathogens on polyurethane, use one cup of bleach to a gallon of water, then rinse.


STORAGE: Leather should be stored in a dry place to prevent mildew. Store polyurethane away from heat greater than 280° or open flame.

Rev 4/20/16

HUMANE JACKETS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrain a person's arms while still allowing the person to walk. Use only as specified in the instructions.

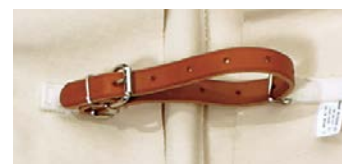

Leather straps may be completely removed from jacket by pulling strap through second keeper on back.

INSTRUCTIONS: Make sure the belts on the back of the jacket are unbuckled. Put the jacket on so that the belts are in back. Pull the short leather straps through the corresponding loops on the other side of the back of the jacket, pull tight, and loop the end of the strap back

through the buckle and keeper. Use this same procedure for all three back straps. Then cross the arms in front and use the arm strap to attach the sleeves to each other in the back using the grommet holes. **NOTE:** Do not use for prolonged periods of time.

	INVENTORY ID	APPROX. JACKET SIZE
Small	CSJR-10S-CS	36-42
Medium	CSJR-10M-CS	42-44
Large	CSJR-10L-CS	44-46
X-Large	CSJR-10XL-CS	48-50

WEIGHT: All jackets weigh approx. 4 lbs.

DESCRIPTION: Humane Jackets come with one 48"-long by 7/8"-wide leather strap to secure the arms and three 22"-long by 7/8"-wide leather straps to secure the body of the jacket.

SAFETY CHECK: Including, but not limited to, cotton duck, stitching, leather, grommets, and buckles.

MATERIALS USED: Heavy cotton duck, 12/13 oz leather, steel loops, 1" cotton web, brass grommets, 1 1/2" press tape, and crotch strap with 1 1/4" cotton web and spring buckle.

CLEANING AND MAINTAINING: For the leather straps: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil. The jacket without the leather straps may be laundered in your facility or dry cleaned.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/8/05

HUMANE SAFETY BLANKET

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: The blanket is designed to be a rugged yet comfortable bed or body covering.



INSTRUCTIONS: Unfold the blanket and use as necessary.

INVENTORY ID: HSB-100

WEIGHT: Approx. 4 lbs.

DESCRIPTION: Approx. 85" by 60" blanket is quilted and hemmed.

SAFETY CHECK: Including, but not limited to, stitching and quilted material.

MATERIALS USED: 1" box quilted quadlan-batting-quadlan and thread.

CLEANING AND MAINTAINING: The blanket can be dry cleaned or cleaned in your facility's laundry.

STORAGE: Store in a dry area to prevent mildew.

HUMANE SAFETY SMOCK

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INSTRUCTIONS: Open the torso hook and loop, and place arms through the armholes. Secure the hook and loop along the torso. Adjust the shoulder straps to custom fit the individual.

The safety smock is versatile and can be worn with the hook and loop closures either in the front or back of the body.

NOTE: It is not necessary to remove handcuffs or other restraints before applying the Humane Safety Smock.

INTENDED USE: The smock is designed to be a rugged yet comfortable modesty covering.



Modesty tab may be used to give a legged appearance for additional modesty covering.

INVENTORY ID: HSS-100

WEIGHT: Approx. 4 lbs.

DESCRIPTION: The smock has hook and loop along the body and on the shoulders for maximum adjustment. All edges are hemmed, not bound.

SAFETY CHECK: Stitching, hook and loop and quilted materials.

MATERIALS USED: 1" box quilted quadlan-batting-quadlan, thread, 4" hook and 4" loop.

CLEANING AND MAINTAINING: The blanket can be dry cleaned or cleaned in your facility's laundry.





STORAGE: Store in a dry area to prevent mildew.

NYLON CONTROL STRAP

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To control movement of restrained person's legs to preserve the integrity of the automobile as well as the protection of all passengers. Use only as specified in the instructions.

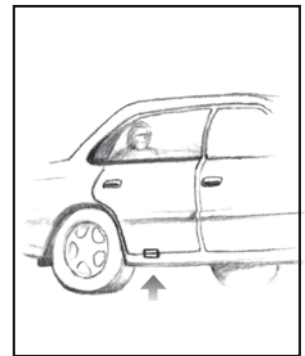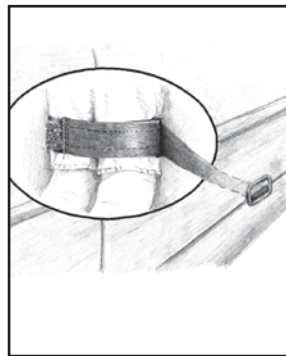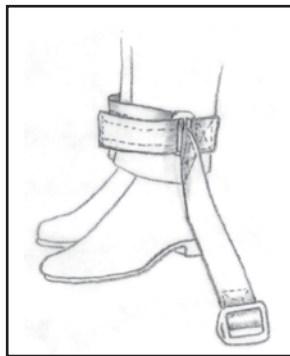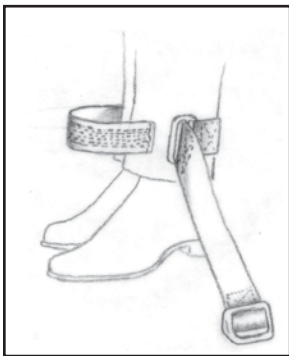

INSTRUCTIONS: Fully extend the nylon strap, wrap it around the ankles with hook and loop side out, and feed through the slide bar. Pull tight around the ankles.

Extend control strap through car door, and close car door on the strap with the trailing slide bar outside the car door.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: NCS-300

WEIGHT: approx. 5 oz.

DESCRIPTION: 56" long overall and 2" wide; loop 16–27" long, and control strap 18" long.

SAFETY CHECK: Including, but not limited to, hook and loop, nylon edges, stitching, and slide bars.

MATERIALS USED: 2" black nylon, 1 ½" black hook and loop, 2" rubber-coated steel slide bars.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. Use a lint brush to clean the hook and loop. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.





Rev 2/20/05

NYLON CONTROL STRAP

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To control movement of restrained person's legs to preserve the integrity of the automobile as well as the protection of all passengers. Use only as specified in the instructions.

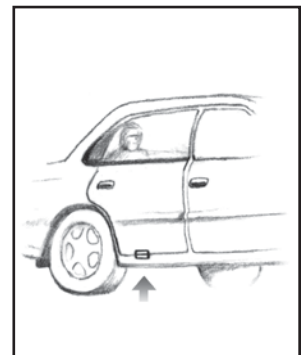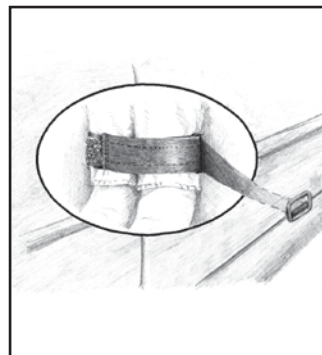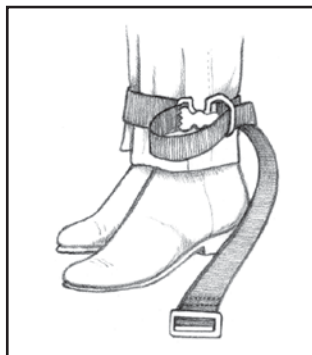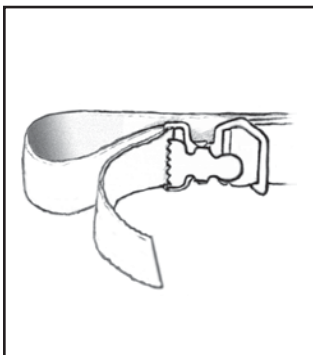

INSTRUCTIONS: Fully extend the nylon strap and wrap it around the ankles. Slide the end through the buckle from behind the teeth, pull to adjust sizing, then slide the end through slot on the other end of the buckle to secure.

Extend control strap through car door, and close car door on the strap with the trailing slide bar outside the car door.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: NCS-900

WEIGHT: approx. 5 oz.

DESCRIPTION: 53" long overall and 1½" wide; control strap 18" long.

SAFETY CHECK: Including, but not limited to, nylon edges, stitching, and slide bar.

MATERIALS USED: 1" nylon, 1" rubber-coated steel slide bar, 1" steel spring buckle.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating blood-borne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 2/20/05

HUMANE BLANKET WRAP

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: This special wrap is designed to transport a person in a horizontal position by using the handles on the side of the wrap. The wrap should go

around the limbs and the body to prevent flailing of the limbs for patient and staff safety.



INSTRUCTIONS: Pull apart the ends of the wrap, detaching the hook and loop closure. Decide how the wrap will be best positioned, wrap around in a safe, secure manner, and align the hook and loop together. Be

sure the handles are in a correct position so the person may be transported properly.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPROX. WEIGHT
HBW-050	72" x 60"	5 lbs
HBW-100	60" x 48"	4 lbs 11 oz
HBW-200	48" x 44"	4 lbs

DESCRIPTION: Each wrap has 12–14" of hook and loop overlap and six handles.

SAFETY CHECK: Including, but not limited to, stitching, hook and loop, press tape, and cotton webbing.

MATERIALS USED: Two-ply cotton webbing, 1 ½" press tape, hook and loop.

CLEANING AND MAINTAINING: All wraps can be dry cleaned or cleaned in your facility's laundry at less than 280 degrees F. with hook and loop in closed position. Hook and loop may be cleaned with a lint brush. When decontamination bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store with hook and loop in closed position in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/8/05

HUMANE WRAPS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: These wraps are designed to aid in physical holds or for transport situations. They are very versatile and may be used around the chest, waist, or legs to prevent flailing of the limbs.



INSTRUCTIONS: Pull apart the ends of the wrap, detaching the hook and loop closure. Decide how the wrap will be best used, wrap around in a safe, secure

manner, and align the hook and loop together.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

Inventory ID	Length	Hook & Loop	Appx. Weight
HWL-050	72" x 8"	24"	16 oz
HWL-100	60" x 8"	23"	13 oz
HWS-200	36" x 8"	14"	10 oz

Other lengths are available upon request.

SAFETY CHECK: Including, but not limited to, stitching, hook and loop, and cotton webbing.

MATERIALS USED: Heavy two-ply cotton webbing, hook and loop.

CLEANING AND MAINTAINING: All wraps can be dry cleaned or cleaned in your facility's laundry at less than 280 degrees F. with hook and loop in closed position. Hook and loop may be cleaned with a lint brush. When decontamination bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store with hook and loop in closed position in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/8/05

RESTRAINT BAGS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To hold restraints in one place while keeping them safe and clean. The small bag can hold a few pair of cuffs and some belts; the large bag can hold multiple pair of cuffs and belts or a set of bed restraints; the leg brace bag can hold one pair of leg braces.

INSTRUCTIONS: Use the zipper to close or open the bag. Do not overfill; this may cause the zipper to malfunction.

INVENTORY ID	APPROX. WEIGHT	SIZE
RBL-100 (Restraint Bag-Large)	9 oz	24" x 12" x 9"
RBS-100 (Restraint Bag-Small)	5 oz	22" x 9" x 9"
CCB-100 (Leg Brace Bag)	5 oz	34" x 14"

SAFETY CHECK: Including, but not limited to, zipper, stitching, and nylon.

MATERIALS USED: 1000 denire nylon cordura and nylon zipper.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating blood-borne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

NYLON ANKLE HOBBLE


IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To allow a restrained person to walk but impedes them from running or kicking. Use only as specified in the instructions.



Standard D-ring closure

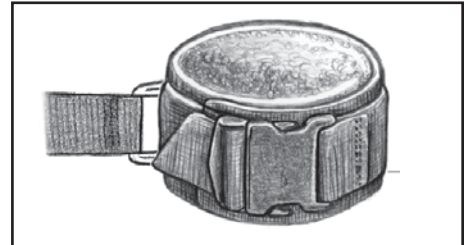

Non-metal style
with side release buckle closure

INSTRUCTIONS: Before applying the restraint, open completely. Wrap the first cuff around the ankle so the D-rings are to the front and the hook end is around the back. Adjust to the proper fitting and secure the hook to the loop.

Feed the 1" strap through the D-rings, then split the D-rings and go back through the second D-ring. Pull on the end of the strap to secure. Use the same procedure for the other ankle.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: NAH-200

WEIGHT: Approx. 7 oz

DESCRIPTION: Ankle cuffs are 15" long and 2" wide. Cuffs are secure first with hook and loop, then a secondary closure of 1" double-D-rings. The anklets are permanently attached to each other by a 9" long by 1" wide nylon strap which is sewn.

SAFETY CHECK: Including, but not limited to, stitching, nylon, and hardware.

MATERIALS USED: 1" and 2" black nylon, 1½" hook and sheet loop, 1" steel D-rings or 1" side release buckle, 1" tuck loop, neoprene.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 4/8/05

DISPOSABLE FLEECE LINERS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Disposable cuff liners will assist in the prevention of chaffing and add to patient comfort.



INSTRUCTIONS: Remove a pair of liners from the box. Be sure you have ankle liners for ankle cuffs and wrist liners for wrist cuffs. Take the pair of liners out of the bag and lay them flat with the elastic strips side up. Slide the restraint cuff between the fleece backing and the elastic strips. The liners should fit so that all size adjustments are accessible and the elastic closest to the closure is on the very end of the cuff.



Restraints sold separately

Inventory ID	Restraint Type	Size	Weight (12 pr. box)
FLEECE LINERS, WRIST			
FWL-200	Leather & Polyurethane	. . 7" x 3" 1 lb.
FLEECE LINERS, ANKLE			
FAL-200	Leather & Polyurethane	. . 9" x 3" 1 lb.


SAFETY CHECK: Including, but not limited to, sewing and the edges of the fleece.

MATERIALS USED: Fleece and spandex.

STORAGE: Store in a dry area to prevent mildew.

DURAMAX® BED

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

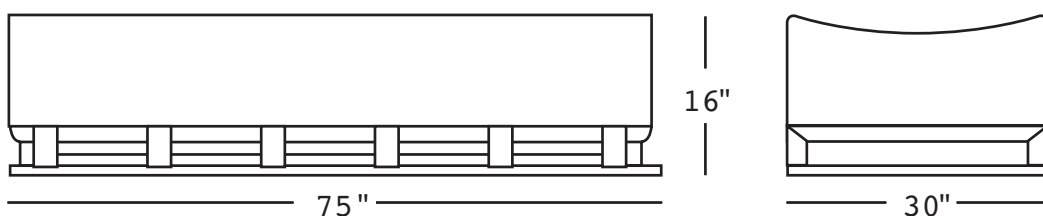

INVENTORY ID: GDB-100

HEIGHT: 16"

WIDTH: 30"

LENGTH: 75"

WEIGHT: 90 lbs.

DESCRIPTION: The bed is molded in one piece and enclosed on all four sides. It has a curved top for added comfort and a slightly textured surface finish.

It can be permanently attached through the recessed base. Two " diameter steel rods are fitted into the base recess and anchored every

10", to allow for attaching restraints along the full length of the bed, according to the person's height.

Please read manufacturer's warning sticker prior to use.

Restraints are sold separately.

MATTRESS PAD

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



DuraMax bed and restraints sold separately.



INTENDED USE: To add to patient comfort.

INVENTORY ID: DBM-100

HEIGHT: 2"

WIDTH: 30"

LENGTH: 75"


WEIGHT: 19 lbs.

DESCRIPTION: The Mattress Pad is one piece of foam enclosed on all four sides. Fits Duramax Bed GDB-100 and Isolation Bed GBIB-100.

CLEANING: Clean with soap and water or bleach and water solution.
Please read manufacturer's warning sticker prior to use.

ISOLATION BED

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

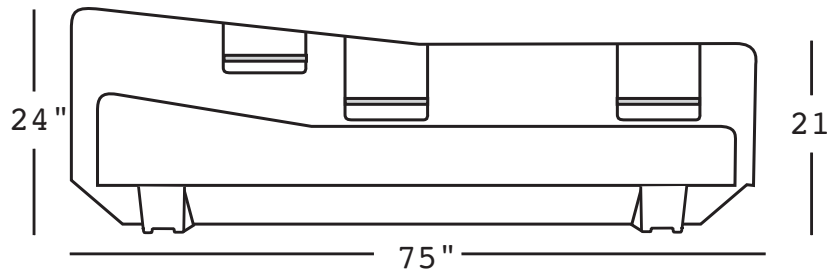**INVENTORY ID:**

Isolation Bed: GBIB-100

Foam Mattress Pad: DBM-100

HEIGHT: 21" at foot; 24" at head

WIDTH: 30"

LENGTH: 75"

WEIGHT: 110 lbs.

DESCRIPTION: The heavy duty Isolation bed includes three bars on each side to attach restraints, slight elevation for the upper body, and is non-porous for easy cleaning.

The bed is molded in one piece and enclosed on all four sides. It has a curved top for added comfort and a slightly textured surface finish.

It can be permanently attached through the recessed base.

Clean with soap and water or bleach and water solution.

Please read manufacturer's warning sticker prior to use.

Fire resistant mattress and restraint systems are sold separately.




PRODUCT SPECS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

LEATHER NON-LOCKING RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.

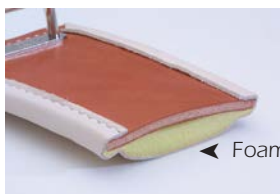

Cross Section

Available with or without inner foam padding for extra comfort.



Shown with belt (sold separately).



INSTRUCTIONS: Open restraint completely and lay flat. Put the limb which is to be restrained on the flattened restraint, roll the loop end of the restraint around the limb, then roll the adjustment end over the top of the loop end. Adjust to the proper fitting, allow the loop to go through the appropriate size adjustment, and

secure by pulling the tail end of a Humane Restraint belt (locking or non-locking) through the loop and around a stationary object (belt sold separately).

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

Adjustments are inside circumference in inches and are in increments of $\frac{3}{4}$ ".

	WRIST		ANKLE		X-LARGE ANKLE	
	Inventory ID	Approx. Weight	Inventory ID	Approx. Weight	Inventory ID	Approx. Weight
Size Adjustments:	(Six) 5 $\frac{3}{4}$ "–9 $\frac{1}{2}$ "		(Six) 8"–11 $\frac{3}{4}$ "		(Six) 11"–14 $\frac{3}{4}$ "	
Standard	WJ-201	9 oz	AJ-201	10 oz	AJ-201XL	11 oz
Foam Padding	WJL-201	10 oz	AJL-201	11 oz	AJL-201XL	12 oz

DESCRIPTION: Wristlets are 12" long and 3" wide, anklets are 14" long and 3" wide, and extra large anklets are 17" long and 3" wide.

SAFETY CHECK: Including, but not limited to, stitching, loops and leather.

MATERIALS USED: 12/13 oz leather, light orthopedic leather lining, steel loops and $\frac{3}{8}$ " poly-foam (foam-lined restraints only).

CLEANING AND MAINTAINING:

For regular cleaning, wipe with a mild soap and warm water. When decontaminating blood-borne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 3/24/16

POLYURETHANE NON-LOCKING RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.

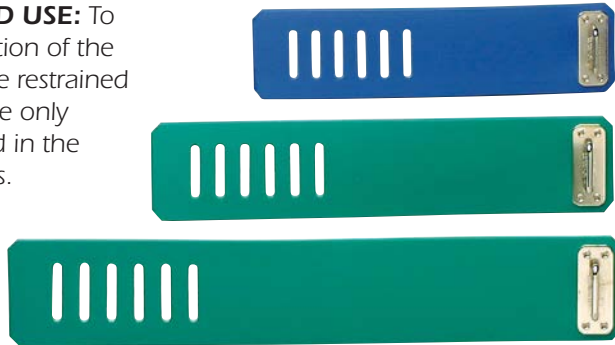

We recommend using this restraint with our disposable fleece liners.

Wrist: FWL-200; Ankle: FAL-200

INSTRUCTIONS: Wristlets and anklets are color coded. For quick visual identification, wristlets are blue and anklets are green. Remember, a person raises their hands to the blue sky and walks on the green grass.

Open restraint completely and lay flat. Put the limb which is to be restrained on the flattened restraint, roll the loop end of the restraint around the limb, then roll the adjustment end over the top of the loop end. Adjust

to the proper fitting, allow the loop to go through the appropriate size adjustment, and secure by pulling the tail end of a Humane Restraint belt (locking or non-locking) through the loop and around a stationary object (belt sold separately).

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

Adjustments are inside circumference in inches and are in increments of $\frac{3}{4}$ ".

	Inventory ID	Approx. Weight	Size Adjustments
Wrist	WJ-201-POLY	9 oz	(Six) 5 $\frac{3}{4}$ "–9 $\frac{1}{2}$ "
Ankle	AJ -201-POLY	10 oz	(Six) 8"–11 $\frac{3}{4}$ "
X-Large Ankle	AJ-201XL-POLY	11 oz	(Six) 11"–14 $\frac{3}{4}$ "

DESCRIPTION: Wristlets are 12" long and 2 $\frac{3}{4}$ " wide, anklets are 14" long and 2 $\frac{3}{4}$ " wide, and extra large anklets are 17" long and 2 $\frac{3}{4}$ " wide.

SAFETY CHECK: Including, but not limited to, hardware and polyurethane.

MATERIALS USED: Polyurethane and steel loops.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/20/16


LEATHER NON-LOCKING RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.



Available with or without inner foam padding for extra comfort.

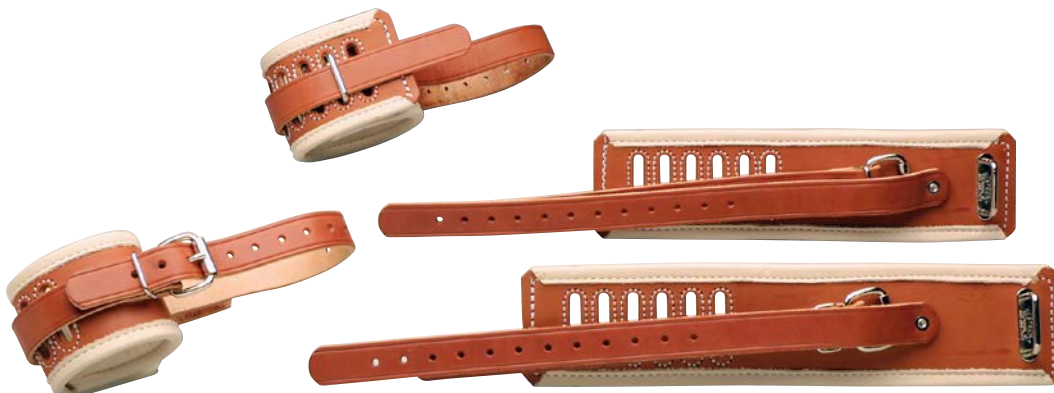

INSTRUCTIONS: Open the restraint by pulling out the leather strap on the wrist or ankle cuff so it is no longer through the loop. Then pull up on the end to release from the loop and lay flat with loop side down.

Put the wrist or ankle on the flattened restraint, wrap the cuff around the limb, and adjust to the appropriate size. Feed the end of the connected strap through the loop to secure. Loop the control strap with the holes around the

rail you want the restrained limb positioned against, feed through the roller buckle, and adjust to the proper fitting. **NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.**

WRIST			ANKLE		
	Inventory ID	Approx. Weight		Inventory ID	Approx. Weight
Standard	W-601	15 oz		A-601	1 lb 2 oz
Foam Padding	WL-601	1 lb		AL-601	1 lb 3 oz

DESCRIPTION: Wrist cuffs are 12" long and 3" wide with six size adjustments ranging from 5¾" – 9½". Ankle cuffs are 14" long and 3" wide with six size adjustments ranging from 8" – 11¾". Control straps are 16" long and 1¼" wide with 12 size adjustments from 6" – 14½".

SAFETY CHECK: Including, but not limited to, loops, stitching, rivets, roller buckles, and leather.

MATERIALS USED: 12/13 oz leather, light orthopedic leather lining, steel loops, steel keepers, and ¾" poly-foam (WL-601 and AL-601 only).

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/8/05

POLYURETHANE NON-LOCKING RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.



INSTRUCTIONS: Open the restraint by pulling out the leather strap on the wrist or ankle cuff so it is no longer through the loop. Then pull up on the end to release from the loop and lay flat with loop side down.



Put the wrist or ankle on the flattened restraint, wrap the cuff around the limb, and adjust to the appropriate size. Feed the end of the connected strap through the loop to secure. Loop the control strap with the holes around the



rail you want the restrained limb positioned against, feed through the roller buckle, and adjust to the proper fitting. **NOTE:** Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	Inventory ID	Approx. Weight
Wrist	W-601-POLY	approx. 1 lb.
Ankle	A-601-POLY	approx. 1 lb 3 oz.

DESCRIPTION: Wrist cuffs are 12" long and 3" wide with six size adjustments ranging from 5¾" – 9½". Ankle cuffs are 14" long and 3" wide with six size adjustments ranging from 8" – 11¾". Control straps are 16" long and 1¼" wide with 12 size adjustments from 6" – 14½".

SAFETY CHECK: Including, but not limited to, loops, rivets, roller buckles, and polyurethane.

MATERIALS USED: polyurethane, steel loops, Steel roller buckle and steel keepers.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.


Rev 4/8/05

LEATHER LOCKING RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Available with or without inner foam padding for extra comfort.

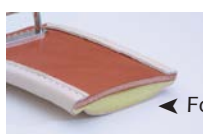

◀ Foam

Cross Section

INTENDED USE: To provide additional security by maintaining the restrained person's limb. The lock promotes authorized release only. The restraint can also be left on the limb without a belt to allow for movement of the limb. Use only as specified in the instructions.

Wristlets and anklets shown with four belts (sold separately).



INSTRUCTIONS: Before applying the restraint, unlock completely. Wrap the cuff around the limb and slide to the appropriate size adjustment. Follow locking instructions on Locks and Keys spec sheet.

Use the same procedure for the other limb. **NOTE:** Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	WRIST		ANKLE		XL ANKLE	
	INVENTORY ID	APPROX. WEIGHT	INVENTORY ID	APPROX. WEIGHT	INVENTORY ID	APPROX. WEIGHT
Standard	WA-501	11 oz	AA-501	14 oz	AA-501-XL	14 oz
Foam Padding	WAL-501	12 oz	AAL-501	15 oz	AAL-501-XL	15 oz

DESCRIPTION: Wrist cuffs are 12" long and 2½" wide, with seven size adjustments. Ankle cuffs are 14" long and 2½" wide, with seven size adjustments. XL ankle cuffs are 17" long and 2½" wide, with seven size adjustments.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, hardware and locks.

MATERIALS USED: 12/13 oz leather, light orthopedic leather lining, steel loops, Humane Restraint L-300 lock buckle, and ¾" polyfoam (WAL-501, AAL-501 and AAL-501-XL only).

CLEANING AND MAINTAINING:

For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 3/24/16

POLYURETHANE LOCKING RESTRAINTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING




We recommend the use of disposable fleece liners with this product.

Wrist: FWL-200
Ankle: FAL-200



INTENDED USE: To provide additional security by maintaining the restrained person's limb. The lock promotes authorized release only. The restraint can also be left on the limb without a belt to allow for movement of the limb. Use only as specified in the instructions.

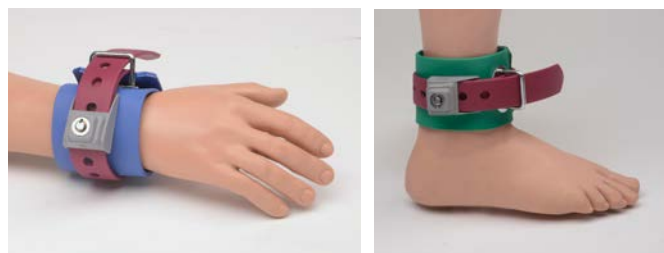

Wristlets and anklets shown with four belts (sold separately).

INSTRUCTIONS: Wristlets and anklets are color coded for quick visual identification. Wristlets are blue and anklets are green. Remember, a person raises their hands to the blue sky and walks on the green grass. Before applying the restraint, unlock completely. Wrap the cuff around the limb and adjust to the appropriate

size adjustment. Follow locking instructions on Locks and Keys spec sheet.

Use the same procedure for the other limb.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPROX. WEIGHT
Wrist	WA-501-POLY	10 oz
Ankle	AA-501-POLY	11 oz
Ankle XL	AA-501-XL-POLY	12 oz

DESCRIPTION: Wrist cuffs are 12" long and 2½" wide, with seven size adjustments. Ankle cuffs are 14" long and 2½" wide, with seven size adjustments. XL ankle cuffs are 17" long and 2½" wide, with seven size adjustments.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, hardware, and locks.

MATERIALS USED: Polyurethane, steel loops, steel lock post, and Humane Restraint L-300 lock buckle.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 3/24/16

LEATHER ANKLE HOBBLE

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To allow a restrained person to walk but prevent them from running or kicking. Use only as specified in the instructions.



Shown with MND-101
Wrist-to-Waist restraint
(sold separately).



Cross Section

Available with or without inner foam padding for extra comfort.



INSTRUCTIONS: Before applying the restraint, unlock completely. Wrap the first cuff around the ankle. Feed the attached strap through the lock. Adjust the strap for the appropriate fitting. Follow locking instructions listed

on lock and key spec sheet. Use the same procedure for the other ankle.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPROX. WEIGHT
Standard	AD-305	1 lb 11 oz
Foam padding	ADL-305	1 lb 12 oz

DESCRIPTION: Ankle cuffs are 14" long and 2½" wide, with seven size adjustments and L-300 locks. The anklets are permanently attached to each other by a 9" long by 1¼" wide polyurethane strap which is double riveted.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, hardware, and locks.

MATERIALS USED: 12/13 oz. leather, light orthopedic leather lining, steel loops, Humane Restraint L-300 lock buckle, and ¾" polyfoam (ADL-305 only).

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/20/16

POLYURETHANE ANKLE HOBBLE

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To allow a restrained person to walk but prevent them from running or kicking. Use only as specified in the instructions.



Shown with
MND-101 Wrist-
to-Waist restraint
(sold separately).



We recommend using this restraint with our disposable fleece liners FAL-200.



INSTRUCTIONS: Before applying the restraint, unlock completely. Wrap the first cuff around the ankle and adjust to the appropriate size adjustment.

Feed the attached strap through the lock. Adjust the

strap and follow locking instructions listed on lock and key spec sheet. Use the same procedure for the other ankle.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: AD-305-POLY

WEIGHT: Approx. 1 lb 9 oz or 12 oz

DESCRIPTION: Ankle cuffs are 15" long and 3" wide, adjustable from 9" to 14". Cuffs securely lock on by a 1 1/4" wide polyurethane strap with L-300 lock buckle. The anklets are permanently attached to each other by a 9" long by 1 1/4" wide polyurethane strap which is riveted.

SAFETY CHECK: Including, but not limited to, stitching, rivets, polyurethane, hardware, and locks.

MATERIALS USED: 1" and 3" wide rolled polyurethane, steel loops, and Humane Restraint L-300 lock buckle.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store in a dry area to prevent mildew.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/20/16

TRANSPORT BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain handcuffed person's wrists at waist by means of the attached cuff or use of the D-ring and a pair of handcuffs. Use only as specified in the instructions.



BACK



FRONT



FRONT

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Pull tail end through roller buckle and adjust to fit snugly. Pull the tail end through the keeper to hold it in place.

Uncuff one arm, feed cuff through the D-ring, and reapply the cuff to secure. OR close attached cuffs on the belt to the detainee.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: MBT-480

WEIGHT: Approx. 1 lb. 9 oz.

DESCRIPTION: 52" long and 1½" wide with 9 size adjustments (30" – 48" by 2" increments).

SAFETY CHECK: Including, but not limited to, stitching, rivets, handcuffs and leather.

MATERIALS USED: 12/13 oz. leather, 1½" steel roller buckle, 1½" steel keeper, pair of handcuffs, and a steel D-ring made to our specifications to fit virtually all handcuffs.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.


NYLON TRANSPORTER

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrain a person's wrists to their waist. The restraint allows no freedom of the arms. The wrist restraints are double-secure with a quick hook and loop closure, then twin D-rings are cinched by a sturdy nylon strap. The restraint secures in back with a side release buckle. Use only as specified in the instructions.



Also available in a non-metal style, with a side release buckle on cuffs.

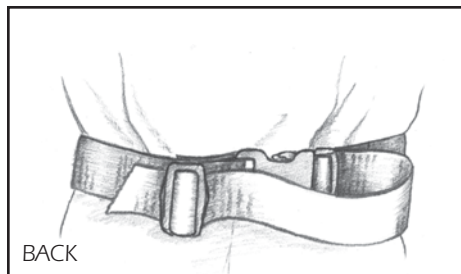

INSTRUCTIONS: Open all closures. Attach the belt around the waist, snap the plastic buckle together, and pull on tail to tighten. Secure by looping the tail back through the slide bar.

Put one wrist into the restraint, close with the hook and loop, and pull the nylon strap through the D-rings to secure. Apply to the other wrist in the same manner.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: NTB-200

WEIGHT: approx. 9 oz

DESCRIPTION: Waist belt is 45" long and 2" wide; wrist cuffs are 12" long and 2" wide.

SAFETY CHECK: Including, but not limited to, stitching, steel D-rings, side release buckle, hook and loop, and nylon.

MATERIALS USED: 2" black nylon, 1½" hook and loop, steel D-rings, 1" black nylon, 1" and 2" side release buckles, 2" plastic slide bar, and neoprene.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating blood-borne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 4/8/05

TRANSPORT BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.

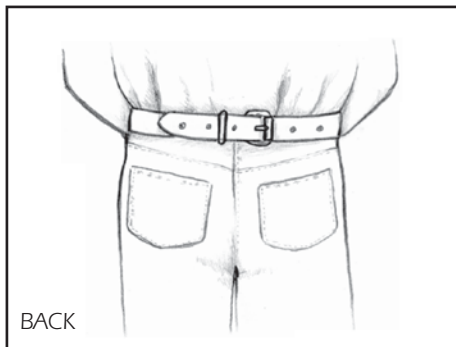

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Pull tail end through roller buckle and adjust to fit snugly. Pull the tail end through the keeper to hold it in place.

Uncuff one arm, feed cuff through the D-ring, and reapply the cuff to secure.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: BT-48

WEIGHT: Approx. 12 oz.

DESCRIPTION: 52" long and 1½" wide with 9 size adjustments (31" – 47" by 2" increments).

SAFETY CHECK: Including, but not limited to, stitching, rivets, and leather.

MATERIALS USED: 12/13 oz. leather, 1½" steel roller buckle, 1½" steel keeper, and a steel D-ring made to our specifications to fit virtually all handcuffs.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.



STORAGE: Store in a dry area to prevent mildew.

LOCKABLE TRANSPORT BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.

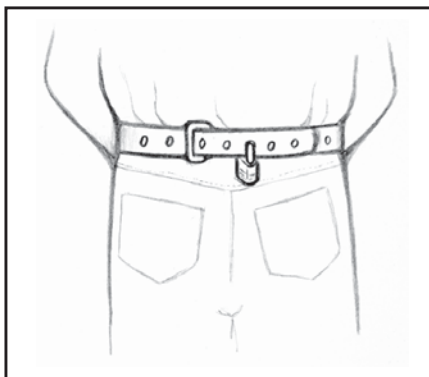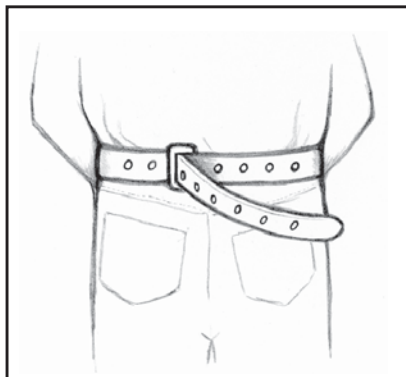

Handcuffs not included.

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Pull tail end through keeper and adjust to fit snugly. Hold in place and lock with padlock.

Uncuff one arm, feed cuff through D-ring, and reapply the cuff to secure.
NOTE: Do not use for prolonged periods of time.

INVENTORY ID: SBT-48-POLY

WEIGHT: Approx. 10 oz.

DESCRIPTION: 54" long and 1 1/2" wide with 9 size adjustments (29" – 48" by 2" increments).

SAFETY CHECK: Including, but not limited to, rivets, handcuff and material fatigue.

MATERIALS USED: Polyurethane, 1 1/2" steel keeper, and a 2" steel D-ring made to our specifications to fit virtually all handcuffs.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water.




STORAGE: Store in a dry area to prevent mildew.

LOCKABLE TRANSPORT BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.

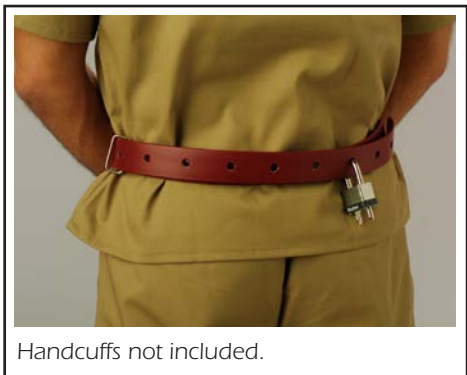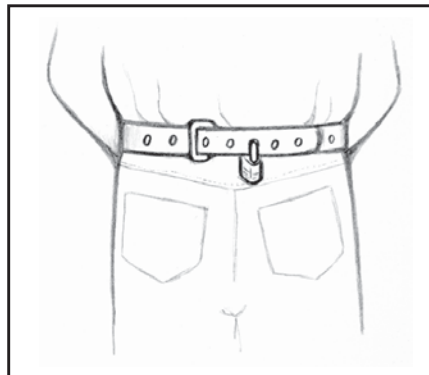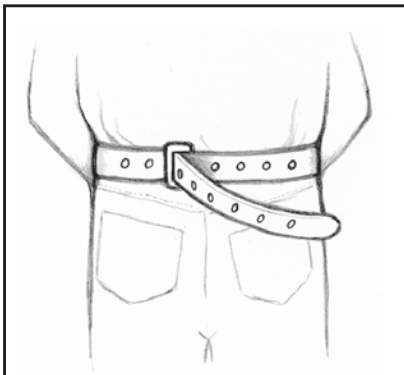

Handcuffs not included.

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Pull tail end through keeper and adjust to fit snugly. Hold in place and lock with padlock.

Uncuff one arm, feed cuff through D-ring, and reapply the cuff to secure.
NOTE: Do not use for prolonged periods of time.

INVENTORY ID: SBT-480-POLY

WEIGHT: Approx. 14 oz.

DESCRIPTION: 54" long and 1 1/2" wide with 9 size adjustments (29" – 48" by 2" increments).

SAFETY CHECK: Including, but not limited to, rivets, handcuff and material fatigue.

MATERIALS USED: Polyurethane, 1 1/2" steel keeper, and a 1 1/2" steel D-ring with attached Smith and Wesson 100 handcuff.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water.



STORAGE: Store in a dry area to prevent mildew.

TRANSPORT BELT WITH HANDCUFF

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.

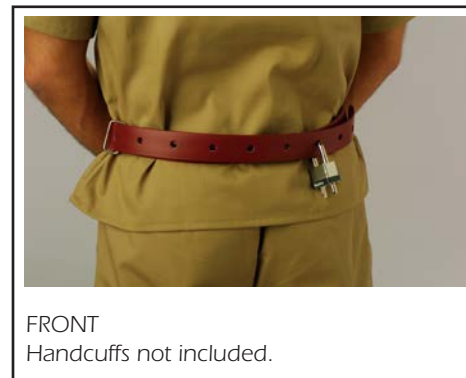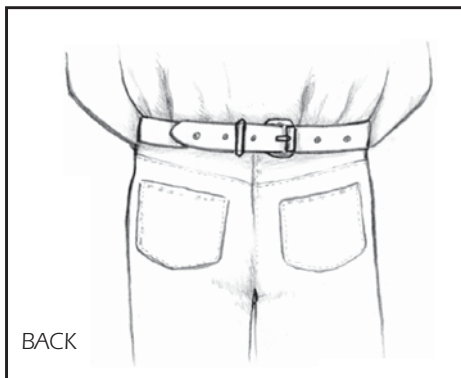

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Pull tail end through roller buckle and adjust to fit snugly. Pull the tail end through the keeper to hold it in place.

Simply close the belt handcuff on the center section of handcuffs on the person.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: CBT-480

WEIGHT: Approx. 1 lb.

DESCRIPTION: 52" long and 1½" wide with 9 size adjustments (31" – 47" by 2" increments).

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather and operation of handcuff.

MATERIALS USED: 12/13 oz. leather, 1½" steel roller buckle, 1½" steel keeper, a 1" steel D-ring and one handcuff.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.


Rev 4/8/05

NYLON TRANSPORT BELT


IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain hand-cuffed person's wrists at waist. Use only as specified in the instructions.

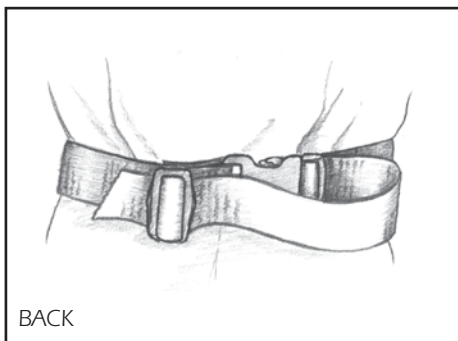

BACK

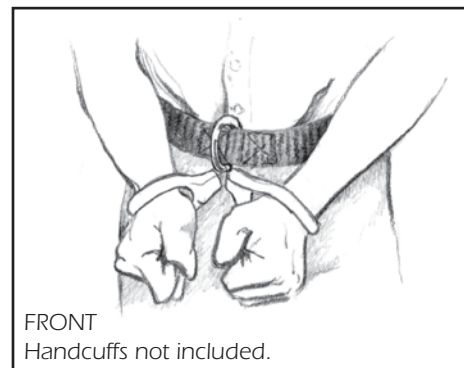

FRONT
Handcuffs not included.

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Snap side release buckle and adjust to fit snugly. Pull the tail end of nylon through the bar slide to keep it from slipping.

Uncuff one arm, feed cuff through the D-ring, and reapply the cuff to secure.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: NT-48

WEIGHT: Approx. 8 oz.

DESCRIPTION: 56" long and 2" wide.

SAFETY CHECK: Including, but not limited to, nylon edges, stitching, and plastic closures.

MATERIALS USED: 2" black nylon, 2" black side release buckle, and a steel D-ring made to our specifications to fit virtually all handcuffs.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating blood-borne pathogens, use bleach while laundering.


STORAGE: Store away from heat greater than 280°F.

NYLON TRANSPORT BELT WITH HANDCUFF


IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain hand-cuffed person's wrists at waist. Use only as specified in the instructions.

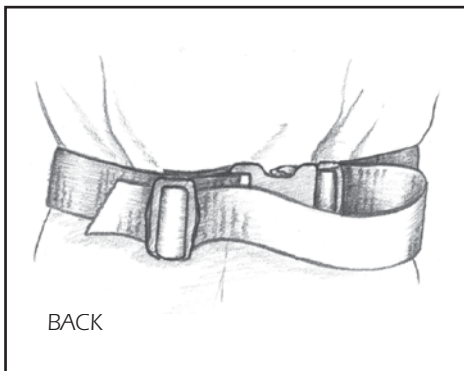

BACK

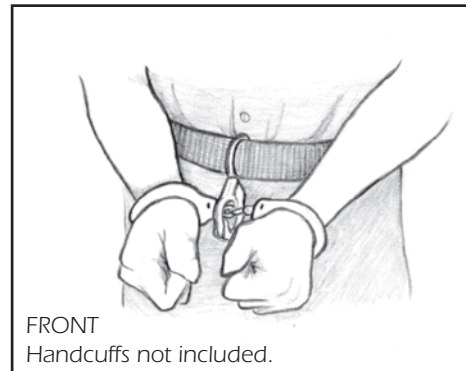

FRONT
Handcuffs not included.

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. Snap side release buckle and adjust to fit snugly. Pull the tail end of nylon through the bar slide to keep it from slipping.

Simply close the belt handcuff on the center section of handcuffs on the person.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: CNT-480

WEIGHT: Approx. 11 oz.

DESCRIPTION: 56" long and 2" wide; side release buckle to close waist belt and handcuff to secure hand-cuffed person to the belt.

SAFETY CHECK: Including, but not limited to, nylon edges, stitching, plastic closures and operation of the handcuff.

MATERIALS USED: 2" black nylon, 2" black side release buckle, a 2" steel D-ring and one handcuff.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating blood-borne pathogens, use bleach while laundering.


STORAGE: Store away from heat greater than 280°F or open flame.

NYLON TRANSPORT BELT


IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.

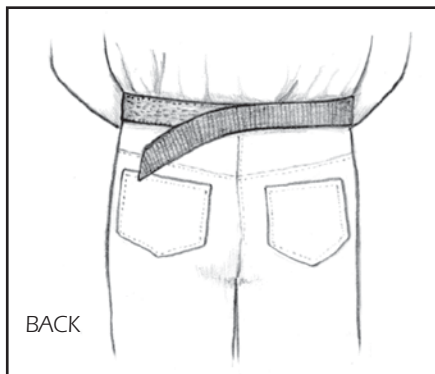

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. The double loop belt section should be placed between the two hook belt sections.

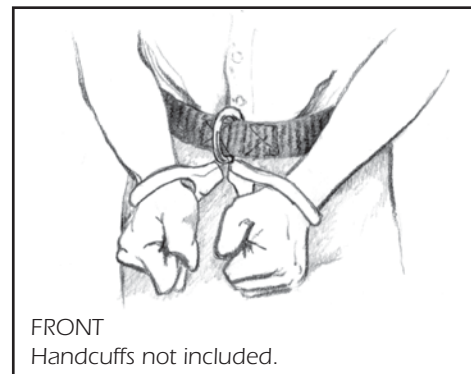

Uncuff one arm, feed cuff through the D-ring, and reapply the cuff to secure.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: HL-48

WEIGHT: Approx. 9 oz.

DESCRIPTION: 58" long and 2" wide with 21" hook and loop overlap closure and a steel D-ring made to our specifications to fit virtually all handcuffs.

SAFETY CHECK: Including, but not limited to, nylon edges, stitching, and hook and loop.

MATERIALS USED: 2" black nylon, hook and loop, and a steel D-ring.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. Use a lint brush to clean the hook and loop. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.


Rev 9/6/02

NYLON TRANSPORT BELT WITH HANDCUFF


IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.

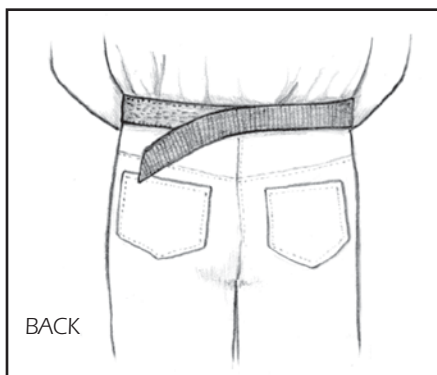

INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. The double loop belt section should be placed between the two hook belt sections.

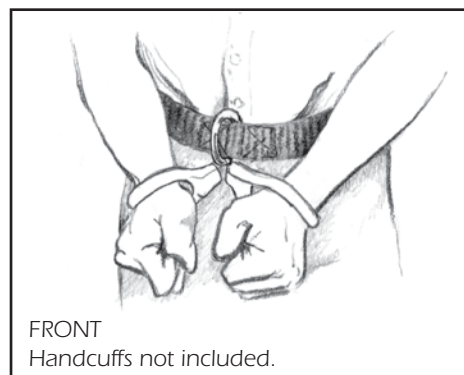

Simply close the belt handcuff on the center section of handcuffs on the person.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: CHL-480

WEIGHT: Approx. 12 oz.

DESCRIPTION: 58" long and 2" wide, with 21" hook and loop overlap closure. Handcuff to secure handcuffed person to the belt.

SAFETY CHECK: Including, but not limited to, nylon edges, stitching, hook and loop and operation of handcuff.

MATERIALS USED: 2" black nylon, hook and loop, a 2" steel D-ring and one handcuff.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. Use a lint brush to clean the hook and loop. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 9/6/02

CUFF AIDE

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Designed to aid law enforcement professionals in maintaining unruly prisoners.



INSTRUCTIONS: Place belt around waist with buckle in back. With the ventilated hand part unsnapped, place handcuffed hands in flat and fold to snap together. Loop hook and loop straps around the folded hands and through the D-rings to secure. Place snap on top D-ring for secondary attachment.

INVENTORY ID: Cuff Aide

WEIGHT: Approx. 10 oz.

DESCRIPTION: 2" wide belt with ventilated hand restraint for cuffed persons.

SAFETY CHECK: Including, but not limited to, belt, buckle, snap and hook and loop.

MATERIALS USED: Hook and loop, nylon, netting, straps, D-ring and buckles.

CLEANING AND MAINTAINING: Can be dry cleaned or cleaned in your facility's laundry at less than 280°F.

STORAGE: Store in a dry area to prevent mildew.

TRANZPORT HOOD

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To reduce the risk of the wearer transmitting fluids (saliva and mucous) from the facial area by spitting, sneezing or coughing. Improper use can result in serious injury or death. **WARNING:** Improper use may cause asphyxiation, suffocation or drowning in one's own fluids.



INSTRUCTIONS: Open bag and remove Hood. Place hood over the head of the person so the mesh fabric is covering the eye area (top) of the head, and the filtration fabric is covering the lower half of the head (below nostrils).

For the best fit, the center elastic will go under the nose and over the ears. For better protection, the elastic may be placed above the nostrils.

Carefully push the white plastic Secure-Lock tab down toward the top of the head while holding the top of the mesh fabric. This should take the slack out of the top and help secure the Hood in position. DO NOT push so tightly as to be uncomfortable or impair vision of the wearer.

CONDITIONS FOR USE:

- Wearer must be under control and restrained.
- Wearer must be under constant visual supervision and should NOT be left unattended.
- DO NOT USE on anyone that is vomiting, having difficulty breathing, or is bleeding profusely from the mouth or nose area.
- Remove all jewelry and eyewear before application.
- The Hood is one size fits all. If there is difficulty applying due to a large size head, discontinue use.

INVENTORY ID:

TranZport Hood, each NIK-7000
Dispenser Box of 40 NIK-7000 Box

WEIGHT: Each 1 oz.
 Box 3 lbs.

DESCRIPTION: One size fits all.

SAFETY CHECK: Including, but not limited to, fabric, mesh, elastic and adjustable tab.

CLEANING AND MAINTAINING: Dispose after use.

STORAGE: Store in a dry area to prevent mildew.

Rev 2/20/05

EMERGENCY RESTRAINT CHAIR

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Now with a brake.

- Wrist and waist restraints lock for added security.
- Fully adjustable wrist, ankle, waist and shoulder restraints.
- Big wheels allow for easy maneuvering on and off elevators and door thresholds.
- Fits through standard doorways.
- Padded surface for subject comfort and protection.
- Solid construction.
- Fits juveniles to adults.

INVENTORY ID: Emergency Restraint Chair


WEIGHT: Approx. 85 lbs.

Rev 4/20/16

BACK SUPPORT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Helps prevent caregiver back injuries by providing back support. Removable suspenders and Sticky Fingers™ maintain placement of back support. Stretchable Spandex allows for a snug yet comfortable fit. Fast application and removal with a premium hook and loop closure. The Model 2000 has a patented V-shaped design for a better fit.

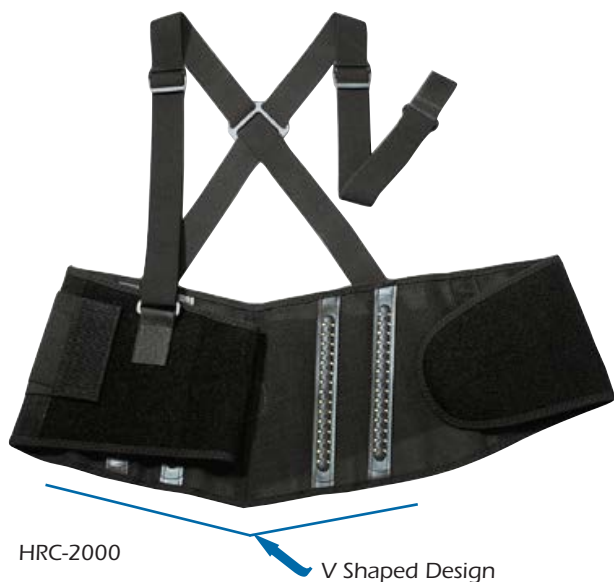

PRODUCT ID	SIZE	WAIST SIZE
HRC-1100	Small	25–30"
	Medium	30–34"
	Large	34–38"
	X-Large	38–42"
HRC-2000	Small	25–30"
	Medium	30–34"
	Large	34–38"
	X-Large	38–42"

PROTECTIVE HELMETS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Hard Shell Helmet
#9821



Soft Shell Helmet
#9829



Face Bar



Face Shield

INTENDED USE: Soft and hard shell protective helmets reduce head injuries. Both styles feature holes for ventilation. Adjustable chin strap with snap closure for fast application and removal. Hard shell helmet can be equipped with either the face bar or shield.

Face Bar and Shield Features:

- Removable face shield and face bar (hard shell helmet only).
- Enhances subject protection and safety.
- Clear Lexan polycarbonate for good visual monitoring.

INVENTORY ID	SIZE	HEAD SIZE	APPX. WEIGHT
9821 Hard Shell Helmet	Small	20¾-21½"	2 lbs.
	Medium	21¾-22½"	2 lbs.
	Large	22¾-23½"	2 lbs.
	X-Large	23¾-25½"	2 lbs.
7630 Face Shield (For Hard Shell Helmet Only)			8 oz.
7636 Face Bar (For Hard Shell Helmet Only)			6 oz.
9829 Soft Shell Helmet	Small	21-21¾"	1 lb. 8 oz.
	Medium	22-22¾"	1 lb. 8 oz.
	Large	23-23¾"	1 lb. 8 oz.
	X-Large	24-24¾"	1 lb. 8 oz.

WEIGHT: Approx. 9 oz.

SAFETY CHECK: Including, but not limited to, snap, strap, and ventilation holes.

CLEANING AND MAINTAINING: Wipe or wash with soap and water or bleach and water.


STORAGE: Store in a dry area to prevent mildew.

Rev 4/20/16

VINYL LIMB HOLDERS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. The lock promotes authorized release only. Use only as specified in the instructions.

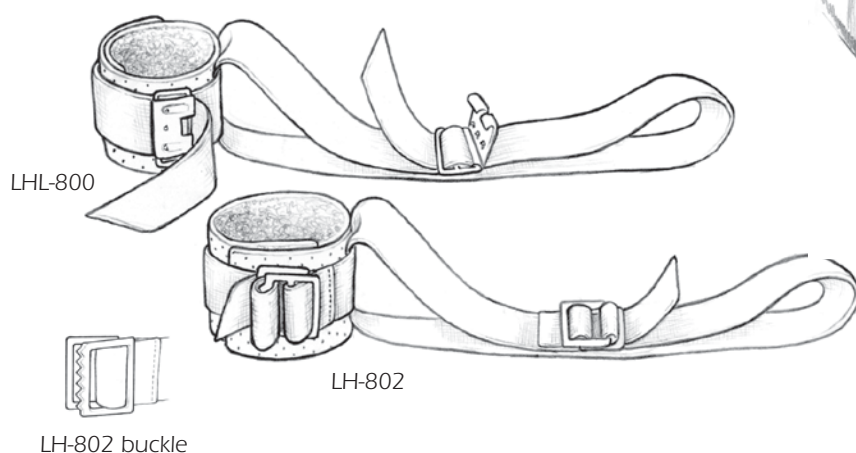

INSTRUCTIONS: Open restraint completely and lay flat. Put the limb which is to be restrained on the flattened restraint and fold the sides over so the hook and loop are attached. Then feed the strap of the secondary closure through the buckle or lock (use locking instructions on back

of this page) and fasten securely. Continue by wrapping the long strap around a bed frame and feeding through the buckle or lock (using locking instructions on back of this page) and fasten securely.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

	INVENTORY ID	APPROX. WEIGHT
L-400 lock	LHL-800	10 oz
Web buckle	LH-802	8 oz

DESCRIPTION: Both cuffs are 12½" long and 2" wide with 4" of hook and loop and fleece lining; secondary closure strap is 19" long and 2" wide; and strap to secure to bed or rail is 46" long and 2" wide. The vinyl is perforated to promote proper air circulation.

SAFETY CHECK: Including, but not limited to stitching, secondary closures, hook and loop, and vinyl.

MATERIALS USED: L-400 lock or web buckle, 2" nylon, 1½" hook and loop, and perforated vinyl with fleece lining.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating blood-borne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 9/6/02


LEATHER UTILITY TORSO BELTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrain a person while in a bed. It may be used with or without wrist and ankle restraints.

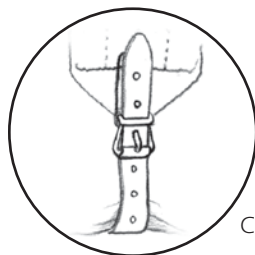


CUR-101

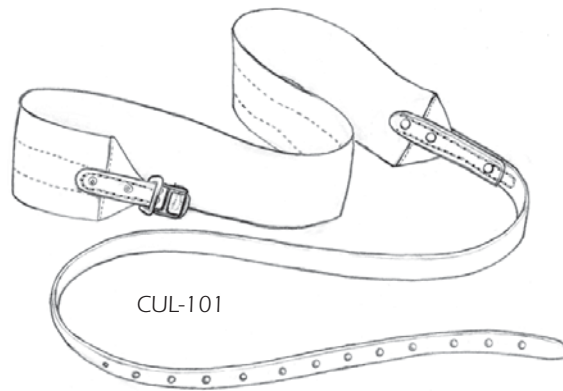

CUL-101



Two torso restraints shown with wrist and ankle bed restraints.

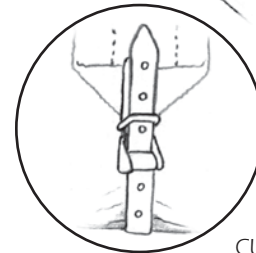

CUL-101

INSTRUCTIONS: To use the CUR-101, lay the restraint over the portion of the body that is to be restrained. If you are using as a chest restraint, lay the restraint over the chest but under the arms. If you are using as a sixth point restraint, lay the restraint above or below the knees. Loop the tail of the leather strap around the bed and person. Feed the tail through the roller buckle, positioning the tongue of the roller buckle in the appropriate size adjustment.

To use the CUL-101, position the restraint as in CUR-101. Loop the tail of the leather strap around the bed and person. Feed the tail into the L-100 lock. Continue by using locking instructions on back of this page.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID	APPROX. WEIGHT
CUR-101 (roller buckles)	1 lb
CUL-101 (L-100 locks)	1 lb 4 oz

DESCRIPTION: Center webbing is 48" long and 4" wide. The side strap is 48" long and 1 1/4" wide. The leather straps have twelve size adjustments. Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, stitching, rivets, leather, webbing, and locks or buckles.

MATERIALS USED: Two-ply cotton webbing; welded steel keepers; 12/13 oz leather; Humane Restraint L-100 lock buckle or roller buckle.

CLEANING AND MAINTAINING: For regular cleaning, wipe the leather with a mild soap and warm water. The webbing can be scrubbed with a soft brush and the same solution. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse. After wiping the leather, allow it to air dry and apply neat's-foot oil.

STORAGE: Store in a dry area to prevent mildew.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 9/6/02



POLYURETHANE UTILITY TORSO BELTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING


INTENDED USE: To restrain a person while in a bed. It may be used with or without wrist and ankle restraints.

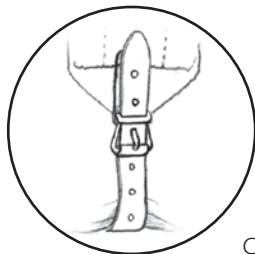

CUR-101-POLY

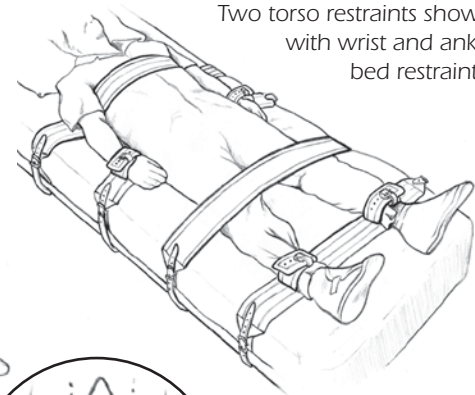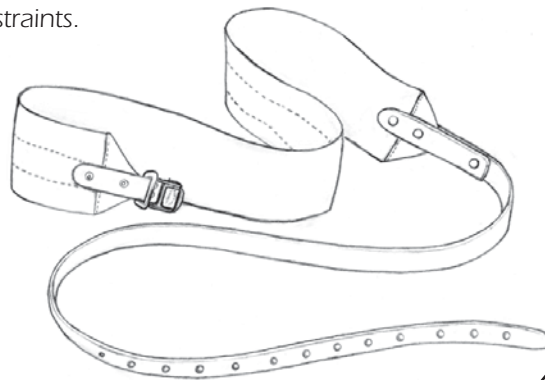

Two torso restraints shown with wrist and ankle bed restraints.

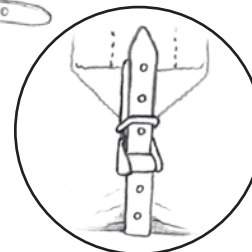

CUL-101-POLY

INSTRUCTIONS: To use the CUR-101-POLY, lay the restraint over the portion of the body that is to be restrained. If you are using as a chest restraint, lay the restraint over the chest but under the arms. If you are using as a sixth point restraint, lay the restraint above or below the knees. Loop the tail of the polyurethane strap around the bed and person. Feed the tail through the roller buckle, positioning the tongue of the roller buckle in the appropriate size adjustment.

To use the CUL-101-POLY, position the restraint as in CUR-101-POLY. Loop the tail of the polyurethane strap around the bed and person. Feed the tail into the L-100 lock. Continue by using locking instructions on back of this page.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID	APPROX. WEIGHT
CUR-101-POLY (roller buckle)	1 lb
CUL-101-POLY (L-100 lock)	1 lb 4 oz

DESCRIPTION: Center webbing is 48" long and 4" wide. The side strap is 48" long and 1 1/4" wide. The polyurethane strap has twelve size adjustments. Custom sizing available upon request.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, webbing, and lock or buckle.

MATERIALS USED: Two-ply cotton webbing, welded steel keeper, polyurethane, and 1 1/4" wide roller buckle or Humane Restraint L-100 lock buckle.

CLEANING AND MAINTAINING: For regular cleaning wipe the polyurethane with a mild soap and warm water. The webbing can be scrubbed with a soft brush and the same solution. When decontaminating blood-borne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.


Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 9/6/02

POLYPROPYLENE TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrain a person while on a safety board. May be used with or without wrist and ankle restraints.

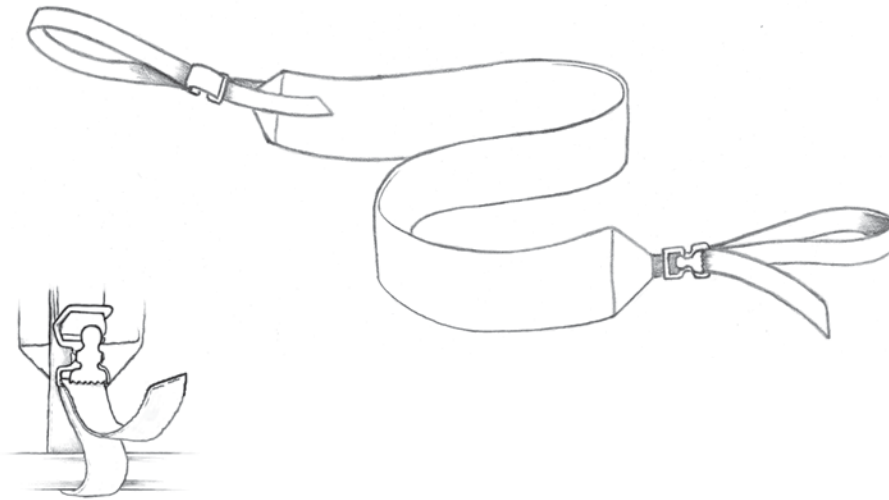

INSTRUCTIONS: To use the nylon torso restraint, lay the restraint over the portion of the body which is to be restrained. If you are using as a chest restraint, lay the longer restraint over the chest but under the arms. Loop the tail of side strap under and around the board frame, and slide through teeth of buckle from below.

Pull to adjust. Slide through other end of buckle to secure. Repeat for other side.

Note: Do not use for prolonged periods of time. Improper adjust meant may prohibit proper circulation.

INVENTORY ID: NBT-101-SP

WEIGHT: approx. 1 lb. each

DESCRIPTION: Polypropylene torso restraint is 85" or 91" long, 1½" wide with a 3" wide center. The 1½" spring buckle is sewn 32" from each end. The torso center is lined with a 2mm padding. Custom sizing is available upon request.

SAFETY CHECK: Including, but not limited to, polypropylene, stitching and spring buckles.

MATERIALS USED: 3" polypropylene, 1½" polypropylene, 1½" spring buckle.

CLEANING AND MAINTAINING: Dry clean or launder at less than 400°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 400°F or open flame.

DELUXE GAIT BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



Deluxe gait belt features multiple handles for team ambulation and assisted movement. Nickel buckle allows for easy sizing and is adjustable from 28" to 53". Four-inch-wide web supports handles and provides additional patient comfort. Available in cotton or polypropylene webbing and both are machine wash and dry.

MATERIAL	PRODUCT ID
Polypropylene . . .	HRC-SD
Cotton.	HRC-CD



PATIENT POSITIONING AID

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

Patient positioning aid with twelve inch handles is a great tool to aid in the movement of patients. Heavy gauge neoprene maximizes patient comfort while minimizing potential caregiver injuries. Machine wash and dry.

PRODUCT ID

Patient Positioning Aid . . .HRC-PPA



HUMANE SLEEPING BAG

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: The sleeping bag is designed to be a rugged yet comfortable bed or body covering.



INSTRUCTIONS: Unfold the sleeping bag and press hook and loop match points together to close.

INVENTORY ID: HSLP-100

WEIGHT: Approx. 4 lbs.

DESCRIPTION: Approx. 85" by 60" sleeping bag is quilted and hemmed, with seven 2x8" hook and loop match points.

SAFETY CHECK: Including, but not limited to, stitching and quilted material.

MATERIALS USED: 1" box quilted Quadlan, thread, hook and loop.

CLEANING AND MAINTAINING: The blanket can be dry cleaned or cleaned in your facility's laundry.

STORAGE: Store in a dry area to prevent mildew.

Rev 8/6/07

HUMANE SAFETY PILLOW/BED ROLL

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: The blanket is designed to be a rugged yet comfortable bed or body covering and may be fashioned into a pillow.



INSTRUCTIONS: Unfold to use as a blanket, or it can be rolled tightly or loosely, then secured with hook and loop to fashion a pillow.

INVENTORY ID: HPW-100

WEIGHT: Approx. 4 lbs.

DESCRIPTION: Approx. 85" by 60" blanket is quilted and hemmed.

SAFETY CHECK: Including, but not limited to, stitching, quilted material, and hook and loop.

MATERIALS USED: 1" box quilted Quadlan, thread, and hook and loop.

CLEANING AND MAINTAINING: The safety pillow/bed roll can be dry cleaned or cleaned in your facility's laundry.

STORAGE: Store in a dry area to prevent mildew.

Rev 8/6/07

ARM RESTRICTOR

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To be used as a fifth point restraint, to limit upper body movement.



INSTRUCTIONS: Open all closures. Secure by looping around arms, then feed tail through slide bar and press hook and loop together. Center strap can be adjusted by moving tail through double slides.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: AAR-100

WEIGHT: Approx. 9 oz.

DESCRIPTION: Center belt is 20" long and 2" wide. Arm cuffs are 16" long and 2" wide.

SAFETY CHECK: Including, but not limited to, stitching, steel slide, hook and loop, and nylon.

MATERIALS USED: 2" black nylon, 1 ½" hook and loop, 2" steel slides.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach when laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 8/6/07

NYLON TETHER

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To limit range of motion. Use only as specified in the instructions.



INSTRUCTIONS: Versatile nylon tether is designed to restrict movement. May be used as a leader/controller for escort, or for cell extractions. Allows for attachment to any kind of door, bar or cuff. 36" tether has loop on one end and snap on other. Do not use for prolonged periods of time.

INVENTORY ID: NTT-100

WEIGHT: Approx. 6 oz.

DESCRIPTION: Overall length 36".

SAFETY CHECK: Including, but not limited to, nylon, stitching, and snap.

MATERIALS USED: 1" nylon and snap buckle.

CLEANING AND MAINTAINING: The tether can be dry cleaned or cleaned in your facility's laundry.

STORAGE: Store in a dry area to prevent mildew.

LOCKS AND KEYS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

L-300 LOCK INSTRUCTIONS:





L-300 Lock




K-300 Key

TO OPEN LOCK: Insert key, turn until lock pops up. Slide out strap. If pressure is on the locking pin, pull strap to release tension.

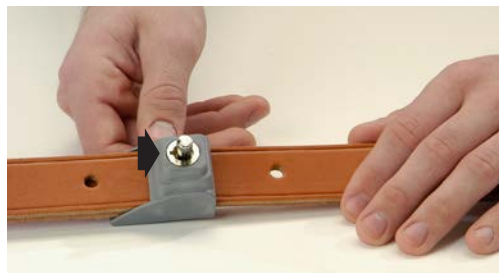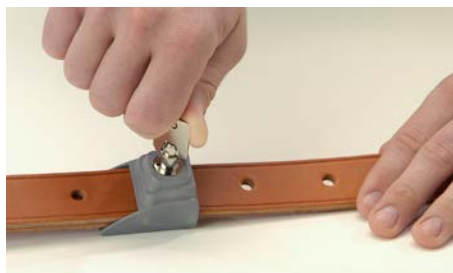

TO CLOSE LOCK: Slide strap through and to adjustment; push down until it clicks.

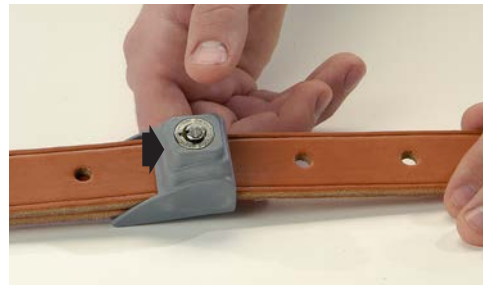

L-400 LOCK INSTRUCTIONS:



L-400 Lock



K-400 Key

All locking restraints and belts include one key per restraint. Extra keys are available for purchase.

TO OPEN LOCK:

Hook the green end of the key around the cross bar from below and with a downward push the hinged plate is flipped up.

TO CLOSE LOCK:

The plain end of the key is hooked over the cross bar, and with a simple upward pull, the hinged plate is snapped shut, gripping the webbing.



Rev 4/8/05

RETURN POLICY: All merchandise must have prior approval before being returned. Merchandise cannot be returned if it has been used or shows signs of use. Custom or special orders cannot be returned for credit. Items found to be ordered in error within two weeks from date of invoice will be exchanged free of charge if resalable. Credit will be applied to customer account. No cash refunds.

WARRANTY: Humane Restraint Co., Inc. will warrant all products for defects in material and workmanship for a period of one year from date of purchase. We will repair or replace, at our option, any product timely delivered to us because of defects in material or workmanship. Humane Restraint Co., Inc. shall not be liable or responsible for incidental or consequential damages.

THE DECISION TO USE RESTRAINTS: Any application of Humane Restraints is solely the decision of the health care professional or law enforcement professional ordering the application of such restraint. Persons deciding to apply Humane Restraints must evaluate the circumstances and accept responsibility for any consequences of such action.

CAUTION: This product may be dangerous if improperly used or applied. This product is not intended for application other than stated or implied. Manufacturer disclaims any warranty or liability for improper usage.

TO ASK QUESTIONS OR GET A COPY
OF OUR CATALOG—800-356-7472.

POLYURETHANE GAIT BELT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: These belts are designed with many uses in mind. They are very versatile and may be used in many situations such as a gait belt.

INSTRUCTIONS: Attach the belt around the waist, and snap the side release buckle together. **NOTE:** Do not use for prolonged periods of time.

INVENTORY ID: HRC-PGB

DESCRIPTION: 2" wide polyurethane belt and side release buckle.

WEIGHT: Approx. weight of belt is 4 oz.

SAFETY CHECK: Including, but not limited to, polyurethane, side release buckle, and polyurethane weld.

MATERIALS USED: Polyurethane and side release buckle.


CLEANING AND MAINTAINING: Simply wipe clean. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store in a dry area to prevent mildew.

LOCKS AND KEYS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

L-100 LOCK INSTRUCTIONS:

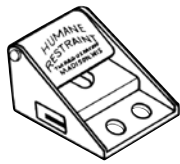

L-100 Lock



K-100 Key


All locking restraints and belts include two keys per lock. Additional keys are available for purchase.

TO OPEN LOCK:


Insert key with large tooth facing left.



Push key inward to open locking "slide" section.

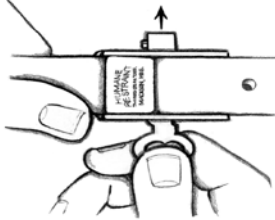

Pull on the end of the belt to loosen locking device.

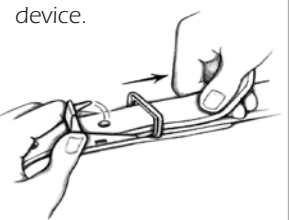


TO CLOSE LOCK:

Insert strap through lock to desired length and close lock into belt hole.




Pull on both straps to tighten lock. Belt should be taut.

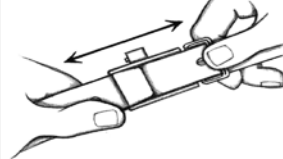

Push in spring-loaded pin and push locking "slide" section into lock.



L-200 LOCK INSTRUCTIONS:

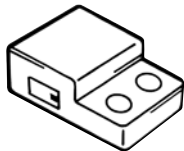


L-200 Lock




K-100 Key

TO OPEN LOCK:

Insert key with large tooth facing left.

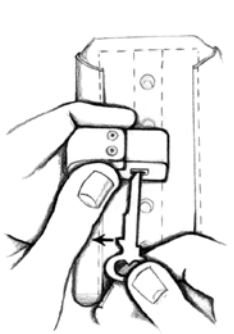


Push key inward to open "slide" locking device.

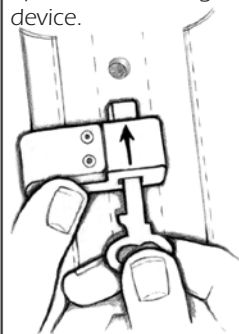


TO CLOSE LOCK:

Push down firmly onto lock stem.

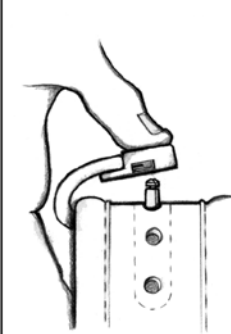

While pressing down, push in "slide" locking section.

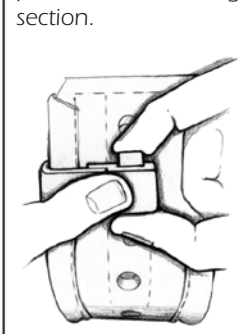

HAND KUZI

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Restricts dexterity to manipulate contraband or other potential hazards.



INSTRUCTIONS: Insert hands into Kuzi and secure handcuffs through D-ring to hold in place around wrists.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID: HRK-100

WEIGHT: Approx. 10 oz.

DESCRIPTION: Kuzi is 10½" deep, 4" in diameter, and made with nylon cordura.

SAFETY CHECK: Including, but not limited to, stitching, nylon cordura, steel D-ring.

MATERIALS USED: 2" nylon cordura, 1" D-ring, heavy duty plastic sheeting.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. and air dry. When decontaminating bloodborne pathogens, use bleach when laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 3/28/16

TRANSPORT BELT WITH HANDCUFFS AND LOCKABLE SLOTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.



INSTRUCTIONS: Wrap the belt around the waist. The double loop belt section should be placed between the two hook belt sections. Then simply close the belt handcuffs on the person.

Secure waist belt with a padlock through slots for extra security.

NOTE: Do not use for prolonged periods of time.



INVENTORY ID: CHL-600-SLOTS

WEIGHT: Approx. 1 lb. 2 oz.

DESCRIPTION: 58" long and 2" wide with 21" hook and loop overlap closure, and handcuffs to secure person to the belt.

SAFETY CHECK: Including, but not limited to, polypropylene edges, stitching, hook and loop, and operation of handcuffs.

MATERIALS USED: 2" black polypropylene, hook and loop, handcuffs.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F.

TRANSPORT BELT WITH LOCKABLE SLOTS FOR DISPOSABLE CUFFS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.



INSTRUCTIONS: Wrap the belt around the waist. The double loop belt section should be placed between the two hook belt sections. Feed disposable cuff through D-Ring and secure wrist.

Secure waist belt with a padlock through slots for extra security.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: HL-DD-SLOTS

WEIGHT: Approx. 9 oz.

DESCRIPTION: 58" long and 2" wide with 21" hook and loop overlap closure, and a steel D-Ring to fit disposable cuffs.

SAFETY CHECK: Including, but not limited to, polypropylene edges, stitching, hook and loop, and D-Ring.

MATERIALS USED: 2" black polypropylene, hook and loop, steel D-Rings.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F.

TRANSPORT BELT WITH HANDCUFF AND LOCKABLE SLOTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.



INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. The double loop belt section should be placed between the two hook belt sections. Then simply close the belt handcuff on the center section of handcuffs on the person.

Secure waist belt with a padlock through slots for extra security.

NOTE: Do not use for prolonged periods of time.



INVENTORY ID: CHL-480-SLOTS

WEIGHT: Approx. 13 oz.

DESCRIPTION: 58" long and 2" wide with 21" hook and loop overlap closure, and handcuff to secure handcuffed person to the belt.

SAFETY CHECK: Including, but not limited to, polypropylene edges, stitching, and hook and loop, and operation of handcuff.

MATERIALS USED: 2" black polypropylene, hook and loop, steel D-Ring, one handcuff.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 5/12/12

TRANSPORT BELT WITH LOCKABLE SLOTS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To maintain handcuffed person's wrists at waist. Use only as specified in the instructions.



INSTRUCTIONS: With restrained person's hands cuffed, wrap the belt around the waist. The double loop belt section should be placed between the two hook belt sections.

Uncuff one arm, feed through the D-Ring, and reapply the cuff to secure. Secure waist belt with a padlock through slots for extra security.

NOTE: Do not use for prolonged periods of time.



INVENTORY ID: HL-48-SLOTS

WEIGHT: Approx. 9 oz.

DESCRIPTION: 58" long and 2" wide with 21" hook and loop overlap closure and a steel D-Ring made to our specifications.

SAFETY CHECK: Including, but not limited to, polypropylene edges, stitching, and hook and loop.

MATERIALS USED: 2" black polypropylene, hook and loop, a steel D-Ring.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.


STORAGE: Store away from heat greater than 280°F or open flame.

Rev 5/12/12

"THE SITTER"

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To assist keeping a detainee in a seated position.

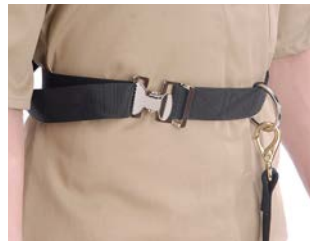

INSTRUCTIONS: Fully open belt by pressing down on spring buckle to release webbing. Loop around waist and pull webbing back through the spring buckle until snug around waist and connect brass eye snap from Humane Restraint Hobble Strap ((HRBS-100).

NOTE: Do not use for prolonged periods of time.

INVENTORY ID: HRSB-100

WEIGHT: Approx. 5 oz.

DESCRIPTION: 64" long and 1 1/2" wide. Strap is fully adjustable and spring buckle makes fast and easy sizing.

SAFETY CHECK: Including, but not limited to, nylon, stitching, and hardware.

MATERIALS USED: 1 1/2" black polypropylene, 1 1/2" spring buckle and D-Ring.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 5/12/12

COMPOUND CUFFS

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To gain quick control of arms. May be used to cuff behind a person's back or to restrain wrists to an anchor point.



INSTRUCTIONS: Fully open loop and pull tabbed end through D-Ring. Secure one wrist by pulling snug. Position other wrist next to secured wrist and wrap webbing.

NOTE: Do not use for prolonged periods of time.

If securing wrist to anchor point, wrap strap around anchor point after securing one wrist.



INVENTORY ID:

HRCC-100 Compound Cuff with Steel D-Ring
HRCC-300 Compound Cuff with Plastic D-Ring

WEIGHT: Approx. 3 oz.

DESCRIPTION: 27" long and 1" wide. Strap is fully adjustable and D-Ring makes fast and easy sizing.

SAFETY CHECK: Including, but not limited to, nylon, stitching, and hardware.

MATERIALS USED: 1" black polypropylene, 1" D-Ring, and hook and loop.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

POLYPROPYLENE LIMB HOLDERS with removable pad

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: To restrict motion of the limbs of the restrained person. Use only as specified in the instructions.



INSTRUCTIONS: Open restraint completely and lay flat. Put the limb which is to be restrained on the flattened restraint and fold the sides over so the hook and loop are attached. Then feed the strap of the secondary closure through the loop and close the strap back onto itself; fasten securely. Continue by wrapping the long strap around an anchor and feeding through the buckle; fasten securely.

NOTE: Do not use for prolonged periods of time. Improper adjustment may prohibit proper circulation.

INVENTORY ID:

PLH-101 Wrist/Ankle Polypropylene Limb Holder

WEIGHT: 7 oz.

DESCRIPTION: Cuffs are 15" long and 2" wide polypropylene. Strap to secure to bed or rail is 36" long and 1" wide.

SAFETY CHECK: Including, but not limited to stitching, spring buckle, and hook and loop.

MATERIALS USED: Spring buckles, 1" and 2" polypropylene, hook and loop.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 5/12/12

HOBBLE STRAP

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Transport: To control movement of restrained person's legs to preserve the integrity of the automobile as well as the protection of all passengers. Escort: To control the movement of legs when applied just above the knees. Use on arms behind back to restrain arm movement. Use only as specified in the instructions.





INSTRUCTIONS: Fully open loop by pressing down on spring buckle to release webbing. Loop around ankles and pull snug. Extend strap through car door and close car door on strap with snap trailing outside. Place above knees for escort situations. The tail may be used to impede unwanted movement. Place on arms behind back for additional control.

NOTE: Do not use for prolonged periods of time.

HRBS-100



HRDR-100

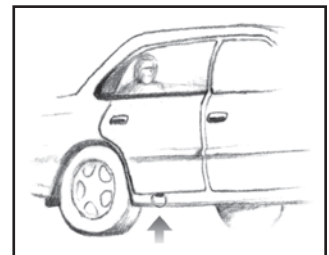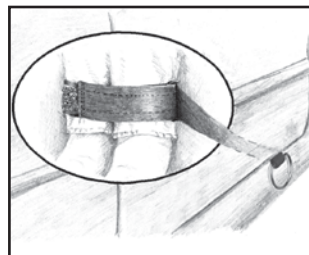

INVENTORY ID:

HRBS-100 Hobbler Strap with Brass Snap

HRDR-100 Hobbler Strap with D-Ring

WEIGHT: 6 oz.

DESCRIPTION: 54" long overall and 1" wide. Strap is fully adjustable and spring buckle makes fast and easy sizing.

SAFETY CHECK: Including, but not limited to, nylon, stitching, and hardware.

MATERIALS USED: 1" black nylon, 1" spring buckle, and #1 brass eye snap hook or 2" D-Ring.

CLEANING AND MAINTAINING: Dry clean or launder at less than 280°F. When decontaminating bloodborne pathogens, use bleach while laundering.

STORAGE: Store away from heat greater than 280°F or open flame.

Rev 5/12/12